

Body Armor® Cast Shoe Cast Boot

Ultralight cast shoe – comfortable fit and protection of the cast

Improved design!



- > rear-entry by flapping the shoe heel backwards – unique bungee closure system patent registered
- > secure hold of the heel
- > toe protection
- > shock absorbing and ultralight EVA material
- > stain resistant, hygienic surface for easy cleaning
- > washable
- > right and left side



DARCO



The Body Armor® Cast Shoe is particularly easy and quick to apply.

- 1 Unhook the bungee-closure system.
- 2 Press the fastener at the bungee-closure system toward the toes in order to widen the ankle strap.
- 3 Folding the shoe heel backwards helps with inserting the foot into the shoe.
- 4 Pull the ankle strap for closure. Hook the closure system again.
- 5 Ready to go!



Indications

- > usable as a walking shoe over cast bandages

Purpose

- > convenience – cast protection
- > easy entry and easily fitted to the cast
- > ergonomically designed to facilitate a correct walking pattern
- > slip-resistant stable outersole

Features

- > extremely light weight EVA material
- > easy closure system
- > improved design – better adapts to the foot
- > can be worn on the right and left

SIZE	ITEM-NO.
------	----------

Color: Dark grey

XS	24.0 cm	CB0
S	25.5 cm	CB1
M	27.0 cm	CB2
L	29.0 cm	CB3
XL	32.0 cm	CB4
XXL	34.5 cm	CB5

Now with a size extra for a more precise size definition.

