



# The original Puzzle Insole® by DARCO – for targeted pressure relief.

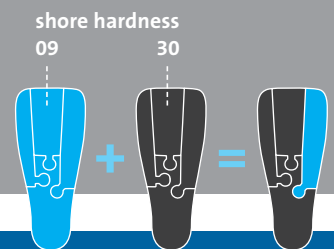

- > multi-zone, multi-indication pressure relief
- > absorbs shock
- > designed for MedSurg™ and MedSurg™ Pro
- > universal left / right fit

Turn two into one – to customize!

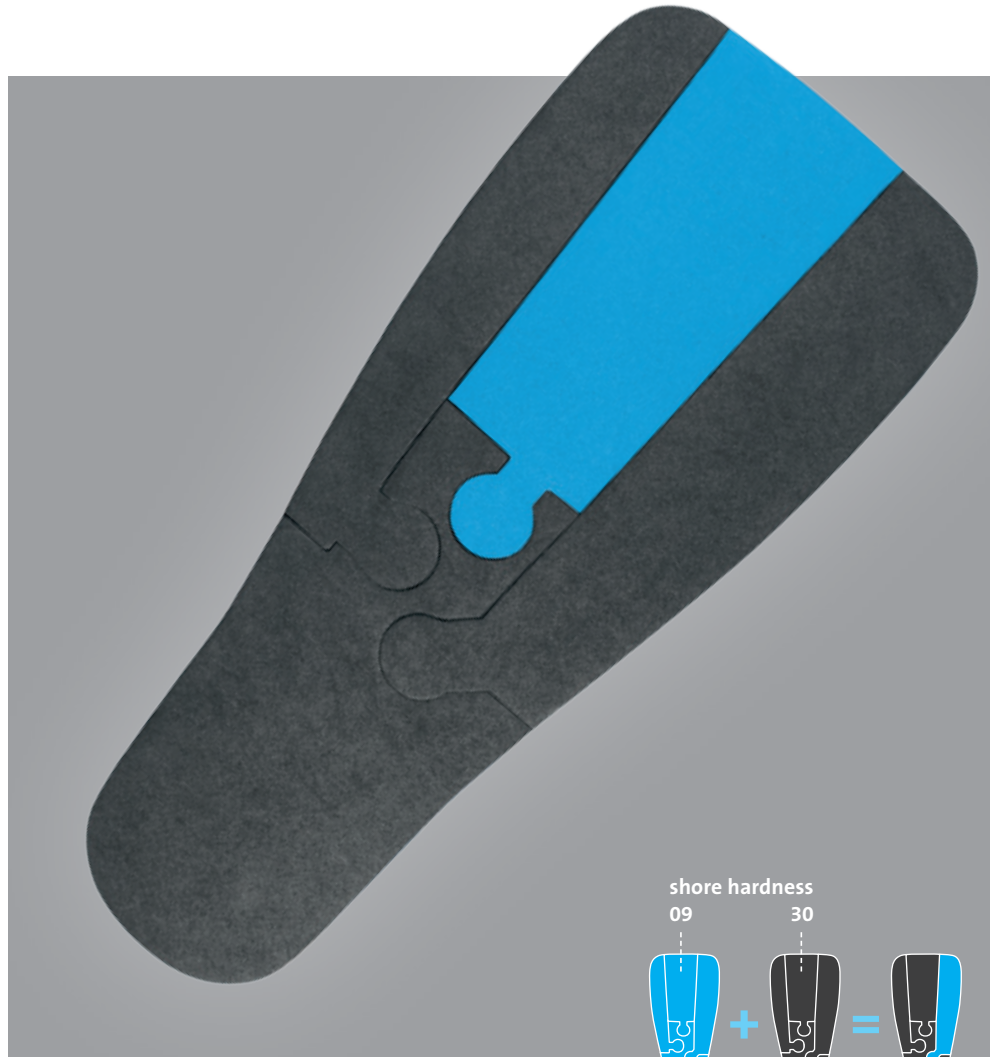

# DARCO

The modular insole was developed for plantar pressure redistribution with specific use after forefoot and midfoot operations.



MT-V-Module (small)

- > Taylor's Bunion (Digitus Superductus)
- > Jones Fracture (Puzzle module ends at the Lisfranc's joint line)



Great Toe Module (big)

- > MTPI-Arthrodesis
- > Austin-, Scarf- and Akin-Osteotomy
- > Closing- and Opening-Wedge-Osteotomy
- > Crescentic-Osteotomy
- > Lapidus-Arthrodesis (TMT1-Arthrodesis)
- > Youngswick-Osteotomy (Puzzle module ends at Cuneiforme I)



Centerpiece

- > Weil- and /or Helal-Osteotomy I-IV
- > Morton's Neuroma
- > Hammer toe Procedures
- > PIP-Arthrodesis
- > Metatarsal Fractures II-IV and PIP-Arthrodesis
- > Shortening-Osteotomies Metatarsals II-IV (Puzzle module ends at the proximal end of Metatarsals II-IV)

By simply replacing the interchangeable modules, the patient can achieve the perfect pressure distribution for each indication. Additional pressure relief of up to 22% in the front is effected by the wedge-shaped sole that tapers toward the heel. Each Puzzle Insole® system includes a black high-density EVA sole, and a blue lower density sole.

Combining the rigid outsole of our MedSurg™ shoe with the modular puzzle results in a selective, focused plantar pressure relief of up to 55%. The plantar pressure in the MedSurg™ shoe was measured under neutral conditions with its regular insole first, and with the modular insole afterward.




#### MedSurg™ Pro and Puzzle Insole® – Scarf Osteotomy for hallux valgus

High side edge ensures a save positioning of the insole.


#### MedSurg™ Insole

Measurement with the regular sole of the MedSurg™ under neutral conditions.

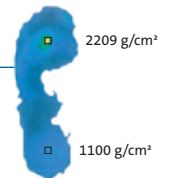



#### Puzzle Insole® without blue mid-module

The wedge-shaped Puzzle Insole® increases the pressure in the heel area.

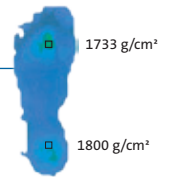

#### Puzzle Insole® with blue mid-module

A selective pressure relief of 45% in the metatarsal area II and III is effected by replacing the mid-module (shore-stiffness 30 / black) with the module featuring less shore-stiffness (09 / blue).

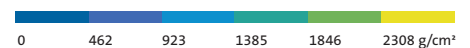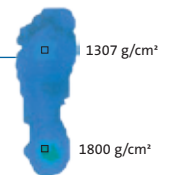

### For MedSurg™ /-Pro

Item-No.      Size (D / EU)

Women:

PI-WS	Small	34.0 – 36.5
PI-WM	Medium	37.0 – 38.5
PI-WL	Large	39.0 – 41.0

Men:

PI-MS	Small	38.0 – 41.0
PI-MM	Medium	41.5 – 43.0
PI-ML	Large	43.5 – 45.0
PI-MXL	X-Large	45.5 – 47.0



DARCO (Europe) GmbH

Gewerbegebiet 18 | 82399 Raisting | Germany | Telephone +49 (0) 88 07.92 28-0 | Fax -22 | info@darco.de | www.darco-online.com