Alround Shoe®

USA/GB Technical data

**6 Sizes:**  
XS (37 – 38), S (39 – 40), M (41 – 42), L (43 – 44), XL (45 – 46), XXL (47 – 48)

**Colour:** Black

**Material:**  
Outsole: Rubber, EVA  
Insole: EVA  
Upper material: Foam Polyester Mesh  
Lining: Brushed Polyester  
Hook and loop fastener straps: Nylon

**Maintenance:**  
The product does not require maintenance.

**Construction features:**  
The upper is made of breathable nylon mesh that can be stretched and adjusted lengthways and sideways. The outer sole is made of polyurethane with a shock absorbing effect. The sole profile facilitates the rolling off movement. The base of the shoe is reinforced throughout. Other features include the wide opening, individually adjustable fastener, a removable sole as well as weatherproof durable material on the shank and sole.

Cleaning instructions

**Hand washing:**  
> When the shoe is slightly dirty, clean it with a damp cloth or a soft brush.  
> The shoe may not tumbled dry or dried on the heater. Allow the shoe to air dry at moderate temperatures.

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**6 Tamanhos:**  
XS (37 – 38), S (39 – 40), M (41 – 42), L (43 – 44), XL (45 – 46), XXL (47 – 48)

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#### AllRound Shoe® Long-term surgical shoe

The AllRound Shoe® long-term surgical shoe provides stability and protects the foot against bumps and reduces pressure on it following surgery and other types of wounds. The shoe can be worn on the left or right foot.

1. Open the shoe using the hook and loop fastener straps (A, B).
2. Make sure that the foot is positioned securely in the shoe.
3. Close the shoe using the hook and loop fastener straps so that it is comfortable for the patient.

#### Indications

- > Postoperative and other types of wounds
- > Arthritis
- > Foot in risk category 0, 1 or 2 according to the German Diabetes Association Guideline

#### Contraindication:

- > Open fractures

#### Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. If the product is handled incorrectly, its function and optimum protection can no longer be guaranteed. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. It is advisable to wear dressings or socks with the shoe. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This aid is a single-use product.

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