Active Body Armor®

USA/GB Indications

- > Distorsion trauma of the upper ankle joint
- > Insufficiency of the capsules and ligaments of the upper ankle joint
- > Prevention of chronic instability
- > Postoperative protection after ligament suturing/reconstruction

Contraindications:

- > Sensorimotor dysfunctions
- > Open fractures
- > Ulcera cruris

Technical data

Size: S (35 – 39), M (40 – 44), L (45 – 49)
Can be worn on the right or left.

Colour: Black-grey

Material:
Orthosis: Polypropylene, fiberglass, EVA, steel
Soft-pad: Polyethylene, polyester
Hook and loop fastener: Nylon

Maintenance: The product does not require maintenance.

Caution

Please always follow these instructions for use. If side effects occur (e.g. skin swelling, pressure points, circulatory disorders), please inform your physician. It is recommended that dressings or socks are worn. If the product is handled incorrectly, its function and optimum protection can no longer be guaranteed. This aid is intended for single-patient use only.

Cleaning instructions

For cleaning, we recommend to wash the inlays by hand with warm water and mild soap or detergent. The plastic parts of the orthosis may be cleaned with a damp cloth. Avoid damaging the material by closing all hook and loop fasteners before washing the orthosis. Do not dry the orthosis in a dryer or on a radiator. Leave it to dry at room temperature.

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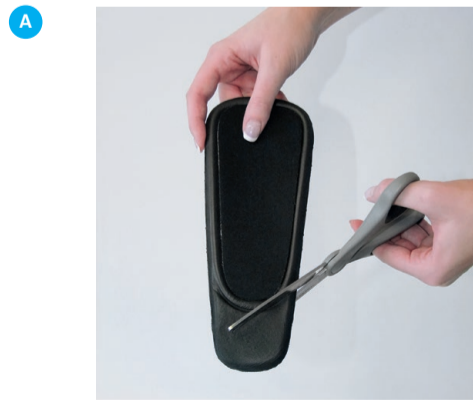
USA / GB Instructions for use

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Body Armor® Active Ankle Brace

The Body Armor® Active is an ankle brace that can be used during the process of rehabilitation as well as preventively. The ankle brace can be worn on the right or left.

- > For maximum stability, we recommend that a sport shoe is worn.
- > For increased comfort, wear a long sock.
- > The Body Armor® Active Ankle Brace is designed for use by one person only and is not meant for multiple use.

Put on the Body Armor® Active as follows:

1. Check whether your shoe has a removable insole.
- 1a. If your shoe has a removable insole or footbed, place the Body Armor® Active underneath the insole or footbed. In this case we recommend to remove the supplied black foam pad for the heel and to not use it.
- 1b. If the insole of your shoe cannot be removed, peel off the adhesive backing of the supplied black foam pad for the heel and fix the pad to the centre of the upper surface of the bottom shell.
2. Open the wide strap to facilitate entry to the orthosis.
3. Insert your Body Armor® Active into the shoe so that the stirrup fits snugly in the shoe. Please make sure that the ankle brace fits closely around your ankle and foot. Position the orthosis sole as close to the back of the shoe as possible. Make sure that the DARCO logo is on the outer side of the foot.
4. Fasten the hook and loop fastener.
5. Then tie your shoes as usual.

Use of the orthosis for preventive care or after rehabilitation:

If you wear the ankle brace for preventive care or if you are able to resume your regular daily activities after an acute injury, you can remodel your Body Armor® Active as follows to achieve a more comfortable fit:

- > Remove the foam cushions from the lateral fixing points.
- > Shorten their length by cutting off the thin lower end at the reshaped edge (image A).
- > Reinsert the cushion after shortening and apply the orthosis as usual.

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