**Body Armor® Sleeve**

Calceñín para fasciitis plantar

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**Body Armor® Sleeve**

Plantar Fasciitis Sleeve

- The opening on which the »Body Armor® Sleeve« logo is embroidered is at the top.
- Pull the sleeve over the foot until the heel is properly positioned in the heel pocket.
- The Body Armor® Sleeve is positioned properly when the heel is securely in the heel pocket.

**Features:**

- > Medical grade fabric
- > Continuous wave compression
- > Special knitting to help prevent pinching and binding
- > Anatomically designed

Can be worn under socks.

**Indications:**

- > Plantar fasciitis
- > Heel Spurs
- > Arch pain
- > Arch weakness
- > Oedema

**Contraindications:**

- > Skin changes (e.g. swelling, redness) due to allergic reactions, inflammation or injury
- > Impaired circulation or lymphatic soft tissue swelling
- > Neurogenic impairments of the sensory function and trophic condition of the skin in the area of the body to be treated (sensory impairment with and without skin damage)
- > Impaired perfusion and sensation in the foot, e.g. diabetes mellitus

**Technical data:**

**Sizes:**  
 Arch circumference in cm  
 S = 17 – 22 cm  
 M = 23 – 27 cm  
 L = 28 – 34 cm  
 Can be worn on the right or left foot.

Where to measure.

Colours: White / Blue

**Material:**  
 81.17% Polyamide  
 18.83% Elastane  
 Latex free

**Maintenance:**

The product does not require maintenance.

**Caution:**

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. The function and optimal protection are no longer guaranteed if the product is incorrectly used. This aid is intended for single-patient use only.

**Cleaning instructions:**

- > Hand wash (cold or lukewarm, max. 40° C)
- > Do not bleach
- > Do not dry in dryer
- > Do not iron
- > Do not dry-clean

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