Body Armor®  
Stirrup Acute

 DARCO (Europe) GmbH  
Gewerbegebiet 18  
82399 Raisting, Germany  
Telefon +49 (0) 88 07.92 28-0 | Fax -22  
info@darco.de, www.darco.de

Made in China

**USA / GB Technical data**

**Sizes:**  
The ankle brace can be worn on the right or left and is available in a single universal size.

**Material:**  
Side stirrups: polypropylene and fiberglass  
Rivets: polypropylene  
Soft inlay: nylon / neoprene laminated together  
Heel pads: nylon  
Straps: polyester  
Bungee closure system: nylon

The brace does not require removal for X-ray as no metal parts are included.

**Maintenance:**  
The product does not require maintenance.

**Caution**

Recommendation to wear bandages or socks underneath the orthosis. Please observe these instructions for use in any case. If you sense any unusual reaction (e. g. skin swelling, pressure points, circulatory disorder etc.) please inform your physician. This aid is intended for single-patient use only.

**Cleaning instructions**

For cleaning we recommend to wash the foam pads by hand with warm water and mild soap or detergent. The plastic parts of the orthosis may be cleaned with a damp cloth. Avoid damaging the material by closing all hook and loop fastener straps before washing the orthosis.

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5.

6.

**Body Armor® Stirrup Acute**  
Ankle joint orthosis

The individual adaption of the Body Armor® Stirrup Acute to the foot will be normally provided after a comparatively short time of wear.

- > For maximum stability we recommend to wear athletic shoes.
- > For advanced comfort please wear an absorptive and highly cut sock.

**Please put on the Body Armor® Stirrup Acute as follows:**

1. Customize the width by removing the hook and loop fastener in the heel area.  
Close it again conveniently according to your individual foot width.
2. Apply the orthosis. The »DARCO«-Logo should be seen at the outside of the foot.
3. Fix the lower hook and loop fastener first.
4. Then fix the upper hook and loop fastener.
5. When applying the orthosis, test for a secure and comfortable fit.
6. Now put on your shoe.

**Indications**

- > for prevention
- > for acute injuries of the ankle joint
- > postoperative rehabilitation
- > for chronic instability of the capsular ligaments

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