Body Armor®
Stirrup Acute

DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Telefon +49 (0) 88 07.92 28-0 | Fax -22
info@darco.de, www.darco.de

Manufactured by:
DARCO Int. Trading (Shanghai) LTD
No. 128 Lane 3458 Hu Min Road
Shanghai 201108, China

USA/GB Technical data

Sizes:

The ankle brace can be worn on the right or left and is available in a single universal size.

Material:

Side stirrups: polypropylene and fiberglass
Rivets: polypropylene
Soft inlay: nylon / neoprene laminated together
Heel pads: nylon
Straps: polyester
Bungee closure system: nylon

The brace does not require removal for X-ray as no metal parts are included.

Maintenance:

The product does not require maintenance.

Caution

Recommendation to wear bandages or socks underneath the orthosis. Please observe these instructions for use in any case. If you sense any unusual reaction (e.g. skin swelling, pressure points, circulatory disorder etc.) please inform your physician.

Cleaning instructions

For cleaning we recommend to wash the foam pads by hand with warm water and mild soap or detergent. The plastic parts of the orthosis may be cleaned with a damp cloth. Avoid damaging the material by closing all velcro® straps before washing the orthosis.

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6.

USA / GB Instructions for use

Body Armor® Stirrup Acute Ankle joint orthosis

The individual adaption of the Body Armor® Stirrup Acute to the foot will be normally provided after a comparatively short time of wear.
> For maximum stability we recommend to wear athletic shoes.
> For advanced comfort please wear an absorptive and highly cut sock.

Please put on the Body Armor® Stirrup Acute as follows:

1. Customize the width by removing the velcro® pads in the heel area. Close it again conveniently according to your individual foot width.
2. Apply the orthosis. The »DARCO«-Logo should be seen at the outside of the foot.
3. Fix the lower velcro® strap first.
4. Then fix the upper velcro® strap.
5. When applying the orthosis, test for a secure and comfortable fit.
6. Now put on your shoe.

Indications

- > for prevention
- > for acute injuries of the ankle joint
- > postoperative rehabilitation
- > for chronic instability of the capsular ligaments

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