# Body Armor<sup>®</sup> Cast Shoe

DARCO International, Inc.  
810 Memorial Blvd.  
Huntington, WV 25701, USA  
Toll-free 800.999.8866  
Phone 304.522.4883  
Fax 304.522.0037  
www.darcointernational.com

DARCO (Europe) GmbH  
Gewerbegebiet 18  
82399 Raisting, Germany  
Telefon +49 (0) 88 07.92 28-0  
Fax +49 (0) 88 07.92 28-22  
info@darco.de, www.darco.de

Made in China

## USA/GB Indications

- > for cast and bandage care

### Technical data

**5 Sizes:**  
XS (24,0 cm), S (25,5 cm), M (27,0 cm), L (29,0 cm), XL (32,0 cm),  
XXL (34,5 cm)

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6.

**Body Armor® Cast Shoe | Cast Boot**

The Body Armor® Cast Shoe is used as a walking shoe over a cast bandage. The shoe fits right and left equally.

1. Unhook the bungee-closure system.
2. Press the fastener at the bungee-closure system toward the toes in order to widen the ankle strap.
3. Folding the shoe heel backwards helps with inserting the foot into the shoe.
4. Pull the ankle strap for closure.
5. Hook the closure system again.
6. Ready to go!

**Caution**

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. If the product is handled incorrectly, its function and optimum protection can no longer be guaranteed. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. It is advisable to wear dressings or socks with the shoe. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This aid is intended for single-patient use only.

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