# Cast Boot Open-Toe + Cast Boot Pediatric

DARCO International, Inc.  
810 Memorial Blvd.  
Huntington, WV 25701, USA  
Toll-free 800.999.8866  
Phone 304.522.4883  
Fax 304.522.0037  
www.darcointernational.com

**CE** **REP**  
DARCO (Europe) GmbH  
Gewerbegebiet 18  
82399 Raisting, Germany  
Telefon +49 (0) 88 07 92 28-0 | Fax -22  
info@darco.de, www.darco.de

Made in China

USA/GB Technical data
<p><b>Sizes:</b> 5 Adult sizes (Cast Boot Open-Toe): XS (33,5 – 37), S (37,5 – 39,5), M (40 – 41,5), L (42,0 – 44,0), XL (44,5 – 47,0)</p> <p>2 Children sizes (Cast Boot Pediatric): XS (up to size 20), S (21 – 22)</p> <p><b>Colours:</b> Adult model (Cast Boot Open-Toe): Blue Children's model (Cast Boot Pediatric): Multi-coloured (fabric design can be varied depending on the manufacture)</p> <p><b>Material (Cast Boot Open-Toe):</b> Outsole: EVA Upper material: Polyester, EVA Lining: Cotton Hook and loop fastener: Polyester</p> <p><b>Material (Cast Boot Pediatric):</b> Outsole: TPR Upper material: 100% Cotton with nylon coating Lining: Cotton Hook and loop fastener: Polyester</p> <p><b>Maintenance:</b> The product does not require maintenance.</p> <p><b>Construction features:</b> The upper consists of cotton fabric with coating. The rubber sole is made of TPR with shock absorbing effect. The sole profile enables an easier rolling-off movement.</p>

Cleaning instructions
<p><b>Hand washing:</b> &gt; When the shoe is slightly dirty, clean it with a damp cloth or a soft brush. &gt; The shoe may not tumbled dry or dried on the heater. Allow the shoe to air dry at moderate temperatures.</p>

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## Cast Boot Open-Toe

Cast Boot Open-Toe / Cast Boot Pediatric

Cast Boot / Pediatric Cast Boot

### Cast Boot Pediatric

Cast Boot Pediatric

## USA/GB Instructions for use

### Cast Boot Open-Toe / Cast Boot Pediatric

Cast Boot / Pediatric Cast Boot

The shoe fits right and left equally.

- Open the boot (A).
- Enter into the opened boot.
- Please ensure that the foot is securely positioned in the boot and that the heel area of the foot is as far back as possible in the boot.
- Close the shoe at the heel and above the instep by means of the hook and loop straps. The foot should be placed comfortably in the boot, not too loose and not too tight.
- The hook and loop strap may now be shortened / cut off individually.

## Indications

- > Fracture and cast care

**Contraindication:**

- > Open fractures

## Caution

**Warning:**

**Caution:**

Your physician has prescribed you /to your child a special shoe of premium quality. This design allows the quickest possible healing of fractures and following surgical intervention, while at the same time maintaining mobility. The special design allows for an easy healing process. Walking with this boot is more difficult than walking with conventional footwear; please read the following precautions.

The shoe provides a specifically rounded sole in order to maintain a physiologically correct gait. Do not attempt to drive while wearing this boot. Use extreme caution when climbing stairs, stepping up onto curbs and walking on uneven surfaces. Walk more slowly than usual. You are not wearing a normal street shoe and more clearance is required for the wedge sole. In case the sole must be fixed due to extensive wearing, please contact your physician or orthopedic technician. Recommendation to wear with bandages or socks. If the product is improperly handled its function and optimal safety can no longer be guaranteed. This aid is intended for single-patient use only.

**Warning:**

**Caution:**

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**Cast Boot Open-Toe / Cast Boot Pediatric**

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