Elbowlift

Walgreen Health Solutions, LLC
1316 Sherman Avenue
Evanston, IL 60201, USA
Phone 001 847.328.9540
info@walgreenhealthsolutions.com
www.walgreenhealthsolutions.com

Emergo Europe B.V.
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Distributor

DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Telefon +49 (0) 88 07.92 28-0 | Fax -22
info@darco.de, www.darco.de

Made in China

USA/GB Technical data

Size:
One size fits all, right and left fit

Colour:
Beige

Material:
Suspension Pad: Latex free polyurethane foam
Hook and loop fastener: Nylon

Maintenance:
The product does not require maintenance.

Construction features:
Elbow suspension pad for off-loading the area around the elbow. Fixation of the pad with dorsally thick foam covered harness.

Cleaning instructions

- > **To hand wash:** Use mild cleanser and warm water.
- > **To machine wash and dry:** Secure straps and place in net laundry bag or pillowcase (60° C).
- > **To sanitize after washing:** Use an antiseptic spray.
- > **To sterilize:** Use standard protocols with a steam autoclave. Allow a few hours to cool and dry completely.

For cleaning you may use water, standard detergents and disinfectants. The straps should be always closed during the process of cleaning. When washed at high temperatures, always allow the Elbowlift to cool and dry before re-opening.

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Elbowlift

Elbowprotection

The Elbowlift Suspension Pad protects the sensitive olecranon from friction burns and contusions.

Application of the elbow suspension pad:

1. Open up the elbow suspension pad by removing the hook and loop fastener of the thick foam covered harness from the suspension padding.
2. Place the patient's elbow onto the suspension padding by positioning the injured part of the elbow to be off-loaded directly onto the center hole with the upper and lower arm comfortably stretched and bedded on the inner side of the suspension pad.
3. Pull the thick foam covered harness over the crook of the elbow and close it laterally by means of the hook and loop fastener. When applying the elbow suspension pad please ensure firm but comfortable placement.

Indications

- > for prevention and off-loading pressure

Contraindications:

- > open fractures

Caution

The elbow suspension pad should be removed every 6 – 8 hours to discover discoloring of the elbow's skin or changing of the blood circulation in time. In case of improper handling, the function and maximum safety of the product can no longer be guaranteed. This aid is intended for single-patient use only.

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