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Distributor
Walgreen Health Solutions, LLC
1316 Sherman Avenue
Evanston, IL 60201, USA
Phone 001 847.328.9540
info@walgreenhealthsolutions.com
www.walgreenhealthsolutions.com

CE
EMERGO Europe B.V.
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Distributor
DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Telefon +49 (0) 88 07.92 28-0 | Fax -22
info@darco.de, www.darco.de

USA / GB	Indications
	<ul style="list-style-type: none"> > For the prevention and treatment of heel pressure ulcers, diabetic foot syndrome and decubitus > Braden scale scores of fourteen or less > Diabetes with neurological sensory deficit > No palpable pedal pulse > Paraplegia > Significant nutritional deficits (serum albumin < 2.2 g/dl) > Prolonged immobility (> hours) or coma
	<p>Instructions for use: Depending on patient / indication either use a Heelift® Glide with smooth foam or convoluted foam.</p>
Caution	
	<p>This is a medical device for care treatment and should only be used under the guidance of your physician or physiotherapist. If you experience pain, abnormal swelling, noticeable changes or other unusual reactions, discontinue use and contact your physician immediately. In case of improper handling function and maximum safety of the product can no longer be guaranteed. This aid is intended for single-patient use only.</p>
Technical data	
	<p>Size: Petite: 15 – 25 cm Calf circumference Standard: 25 – 38 cm Calf circumference Bariatric (only Heelift® Glide Suspension Boot Smooth): 38 – 58 cm Calf circumference</p> <p>Content: Elevation pads, Off-loading boot</p> <p>Latex free polyurethane foam</p>
Cleaning instructions	
	<ul style="list-style-type: none"> > To hand wash: Use mild cleanser and warm water. > To machine wash and dry: Secure straps and place in net laundry bag or pillowcase (60° C). > To sanitize after washing: Use an antiseptic spray. > To sterilize: Use standard protocols with a steam autoclave. Allow a few hours to cool and dry completely. <p>For cleaning you may use water, standard detergents and disinfectants. The straps should be always closed during the process of cleaning. When washed at high temperatures, always allow the Heelift® Glide to cool and dry before re-opening.</p>

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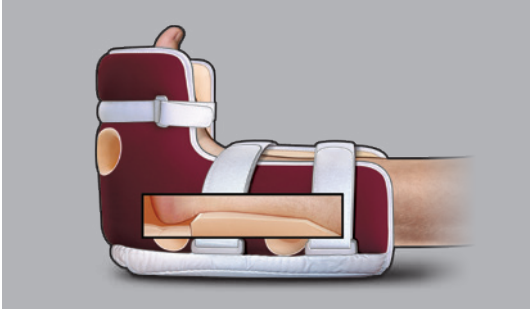
USA/GB Instructions for use

Heelift® Glide Suspension Boot

1. Open up the Heelift® Glide boot. Set the extra pad aside. Keep it available, you may need it later to further customize the boot.
2. Place the foot inside the boot with the heel positioned above the heel suspension opening. The heel should hang over the fixed elevation pad. (Picture 1 + 2) For particularly long or large legs, follow listed sizing guidelines and use the extra pad as directed to customize the fit.
3. Pull the closure straps over the leg towards D-rings. Thread the straps through the D-rings. Use the hook and loop fastener closures to secure the straps (Picture 3). Leave flaps slightly open to provide added ventilation.
4. Test for the proper fit. You should be able to fit your fingers between the heel opening and the bed (Picture 4). The boot should not move freely on the leg. The straps should not be in contact with the skin on the top of the leg (Picture 5).

- Customizing the fit:**
- > **Longer legs:** Place the extra pad slightly more outward than the fixed pad.
 - > **Foot drop:** Secure the extra pad in a vertical position behind the sole of the foot. (Picture A)
 - > **Malleolar decubitus:** Cut away the bumps surrounding the ankle. Cut a portion of the fixed pad if necessary. (Picture B)
 - > **Hip rotation:** Secure the extra pad in a horizontal position to outer side of the boot. (Picture C)
 - > **Bariatric legs:** If the heel comes into contact with the bed sheet, add the extra pad on top of the fixed pad for increased elevation
 - > **Achilles tendon:** If redness occurs in this area, cut a »V« shape out of the fixed pad. If using both pads, customize the upper pad. (Picture D)

Instruction warning / risks

Heel pressure is transferred to the heel cord and gastrosoleus complex. The Heelift® Glide should be removed and the leg and foot inspected for skin breakdown, every 6 – 8 hours. Should redness (erythema) occur, a V-shaped wedge should be removed from the fixed pad, or the spare pad applied proximal to the tendon, or a wedge cut out of the spare pad which can then be applied.

Edematous patients may experience harmless dimpling of the skin using the Heelift® Glide with convoluted foam, and may therefore prefer to choose the Heelift® Glide with smooth foam.

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