**5 Sizes:**  
XS (33,5 – 37), S (37,5 – 39,5), M (40 – 41,5), L (42,0 – 44,0), XL (44,5 – 47,0)

The shoe fits right and left equally.

**Material:**  
Outsole: TPR  
Insole: EVA  
Upper material: Polyester  
Hook and loop fastener: Nylon

**Maintenance:**  
The product does not require maintenance.

**Construction features:**  
The shortened and patent protected sole provides effective pressure off-loading. The removable insole can be exchanged by the PegAssist® Insole.

**Recommended add-ons:**  
TwinShoe®, Balance shoe  
PegAssist® Insole, Individually customizable insole for targeted off-loading

**Cleaning instructions**

**Hand washing:**  
> When the shoe is slightly dirty, clean it with a damp cloth or a soft brush.  
> The shoe may not tumbled dry or dried on the heater. Allow the shoe to air dry at moderate temperatures.

**5 Größen:**  
XS (33,5 – 37), S (37,5 – 39,5), M (40 – 41,5), L (42,0 – 44,0), XL (44,5 – 47,0)

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### HeelWedge®

### USA/GB Instructions for use

**HeelWedge®**  
Shoe to reduce pressure on the heel

- Open the shoe (A).
- Insert the foot into the opened shoe.
- Make sure that the foot is positioned securely in the shoe.
- Close the shoe with the hook and loop fasteners (A). So that the fit is snug, but not too tight.
- Only roll as far as the inner edges (B) when walking. Rolling as far as the outermost edges (C) can have a negative effect on the results of surgery.
- When worn for longer periods, it is recommended that the height of the other foot is adapted to match. The shoe should not be worn for longer than 6 weeks.

### Indications

- > Postoperative healing for either soft tissue or bony structure of the heel
- > Selective off-loading after fractures
- > For the treatment of ulcerations present on the heel area (rear foot ulceration)

#### Contraindication:

- > Open fractures

### Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. It is advisable to wear dressings or socks with the shoe. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This aid is intended for single-patient use only.

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