# MA Spacer (MIS Akin Spacer)

**+**ARCO

### MA Spacer (MIS Akin Spacer)

Toe separator

1. First of all, obtain a clear overview of the indication and fitting.
2. Use the toe separator on clean (washed with warm water and soap), dry skin.
3. Place the separator between the first and second toes. The notch points in a proximal direction; make sure that there is no hollow space between the MA Spacer and the space between the toes.
4. The longer part points in a dorsal direction and the short part is for use in the plantar region.
5. The soft, medical material adapts individually to the contours of the toes.
6. Use the tapes to fix the aid in place.

## Indications

- > postoperative care of minimally invasive hallux valgus surgery (Akin osteotomy without osteosynthesis material)
- > conservative treatment of hallux valgus

### Contraindications:

- > large skin injuries
- > open wounds with risk of infection
- > uncertain diagnoses
- > allergic skin reactions to silicone

## Technical data

**Size:** Universal size | 11 mm | can be worn on the left or right foot

**Color:** White

**Material:** Medical silicone

**Maintenance:** The product does not require maintenance.

## Design features

- > the notch points in a proximal direction and fills the area (space between the toes) between the first and second toes.

## Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. If the product is handled incorrectly, its function and optimum protection can no longer be guaranteed. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. It is advisable to wear dressings. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This aid is intended for single-patient use only.

## Cleaning instructions

The MA Spacer (MIS Akin Spacer) toe separator is washable and can be disinfected, if required. Clean the product with household soap. Do not use solvents. Allow the product to dry thoroughly.

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