MECRON Belt

MECRON
by **DARCO**

USA / GB Instructions for use

MECRON Belt
Thorax and abdominal bandage

The MECRON thorax and abdominal bandage is used for compression and is made from flexible, breathable cotton material. The bandage can be individually closed by attaching the wide hook and loop fastener of the waist band to the cotton fabric. The combination of inelastic and flexible material enables the bandage to be optimally adapted to the anatomical situation.

Recommended use:

- Open the hook and loop fastener strip of the abdominal bandage.
- Wind the bandage around the patient's torso so that it fits his /her body snugly.
- Close the hook and loop fastener over the abdomen.

Attach the bandage in such a way to ensure a snug fit. Ensure not to trap any skin around the hook and loop fasteners. To prevent constriction, do not over-tighten.

Indications

- Postoperative after abdominal surgery
- Umbilical hernias
- Sagging belly
- Redression after a pregnancy
- Scar protection

Contraindications:

- No contraindications are known.

Cleaning instructions

The abdominal bandage is washable at 60°. Close the hook and loop fasteners prior to washing.

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