USA/GB Technical data

Sizes:

The elbow splint comes in 4 sizes depending upon upper arm circumference. Measure the circumference 10 cm above the middle of the elbow with the arm bent 45 – 90 degrees. The splint can be worn on the left or right.

Size	Circumference
S	25 – 30 cm
M	30 – 35 cm
L	35 – 40 cm
XL	40 – 50 cm

Colour: blue vinyl, black

Material:

Outer side of the splint: PVC compound
Inner side of the splint: polyamide
Edging strip: 100% cotton
Velcro fasteners: polyamide

Care: The product is maintenance-free.

Caution

Always follow these instructions for use. If you notice any side effects (e.g. swollen skin, pressure points, disrupted blood circulation, etc.) please consult your doctor. If used improperly, the product's function and optimum protection can no longer be guaranteed.

This appliance is intended to assist you in your recovery and should only be used as prescribed by (or in consultation with) your doctor or medical specialist. DARCO (Europe) GmbH does not assume any responsibility.

Avoid using the splint in water and do not expose it to acids and solvents. It is advisable to wear dressings. We recommend that you refrain from driving while you are wearing this appliance. This appliance is a single-use product.

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MECRON

by **DARCO**

USA/GB Instructions for use
<p>MECRON Elbow Splint Elbow splint</p> <p>The Mecron elbow splint is used to immobilize the elbow joint. The elbow splint features two aluminium flexion rods for stabilization. The rods are accommodated in a bag which has a velcro fastener. The rods are removable and can be re-bent as required. The splint is fixed with velcro fasteners and provides a non-slip fit which is comfortable for patients. The materials used are easy on the skin and breathable. The sidebands are for fixing the pre-bent position in degrees.</p> <p>Recommended use:</p> <ol style="list-style-type: none">Open the velcro strips of the elbow splint. Place the patient's arm in the splint so that the elbow rests comfortably in the pre-defined position in degrees. Close first the top, then the bottom velcro fastener. Now you can close the other two velcro fasteners. <p>Slightly bending the aluminium rods in the splint enables them to be adjusted to the arm's anatomy. Attach the splint in such a way to ensure a snug fit. Ensure not to trap any skin around the velcro fasteners. To prevent constriction, do not over-tighten.</p>
Indications
<p>> Elbow sprain > Elbow osteoarthritis > Fracture after-care > Rheumatic elbow effusions > Elbow dislocation > Epicondylitis</p> <p>Contraindications:</p> <p>> No contraindications are known.</p>
Cleaning instructions
<p>The MECRON elbow splint is washable at 30°. Please ensure that you do not spin the splint. Remove the aluminium rods and close the velcro fasteners prior to washing. After washing, pull the splint into shape, re-insert the rods and place horizontally to dry.</p>

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