Hip-Spica MECRON

USA/GB Technical data

Sizes:
The hip spica bandage comes in 2 sizes depending upon circumference. Measure the hip circumference. The bandage comes in 2 versions for the left and right side, and also double-sided.

Size	Circumference
M	95 – 115 cm
L	105 – 125 cm

Colour: White

Material:
Outer side of the inelastic fabric: polyamide
Inner side of the inelastic fabric: 100% cotton
Waist band: 89% viscose, 11% elastodiene
Yarn: 100% polyester
Edging strip: 100% cotton
Velcro fasteners: polyamide

Care: The product is maintenance-free.

Caution

Always follow these instructions for use. If you notice any side effects (e.g. swollen skin, pressure points, disrupted blood circulation, etc.) please consult your doctor. If used improperly, the product's function and optimum protection can no longer be guaranteed.

This appliance is intended to assist you in your recovery and should only be used as prescribed by (or in consultation with) your doctor or medical specialist. DARCO (Europe) GmbH does not assume any responsibility.

Avoid using the bandage in water and exposure to acids and solvents. It is advisable to wear dressings. We recommend that you refrain from driving while you are wearing this appliance. This appliance is a single-use product.

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USA/GB Instructions for use

MECRON Hip Spica Hip spica bandage

The MECRON hip spica bandage is used for compressing and stabilising the hip joint. The velcro fasteners permit free adjustment both at the thigh and at the hip.

Recommended use:

1. Open the velcro strips of the hip spica bandage.
2. Wind the bandage around the patient's pelvis and thigh so that it fits snugly.
3. First close the velcro fastener at the pelvis.
4. Now you can close the velcro fastener at the thigh.

Attach the bandage in such a way to ensure a snug fit. Ensure not to trap any skin around the velcro fasteners. To prevent constriction, do not over-tighten.

Indications

- > Stabilisation after repositioning a dislocated hip joint
- > Stabilisation after total endoprosthetic replacement
- > Hip injuries
- > Chronic hip instabilities

Contraindications:

- > No contraindications are known.

Cleaning instructions

The MECRON hip spica bandage is washable at 30°. Close the velcro fasteners prior to washing.

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