# MECRON Knee Splints

MECRON Knee  
Clinical

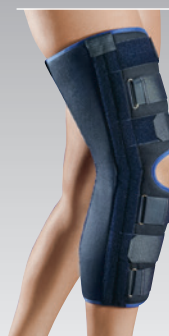
MECRON Knee  
Straight

MECRON Knee  
3-part

MECRON Knee  
3-part Plus

MECRON Knee Flexion

MECRON Knee Pediatric

**DARCO**

**MECRON Knee Clinical**

**MECRON Knee Straight**

**MECRON Knee 3-part**

**MECRON Knee 3-part Plus**

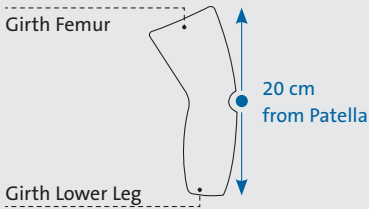
## MECRON Knee Flexion

## MECRON Knee Pediatric

**MECRON**  
*by* **DARCO**

**Girth:**

The 1-part MECRON knee braces need to be ordered in relation to upper and lower leg circumference. Measure the circumference both 20 cm above and below the middle of the patella.

The 3-part MECRON knee braces are individually adjustable to the patient's leg circumference (up to a max. thigh circumference of 74 cm).

**Length:**

For maximum stability, the knee brace should be positioned one handwidth below the groin and should end 1.5 handwidths above the ankle. This applies to 1-part and 3-part knee braces.

**1-part knee braces:**

**MECRON Knee Flexion & MECRON Knee Straight**

Length	Size	Girth Femur	Girth Lower Leg	Colours
40 cm	S	43 cm	30 cm	Jeans Blue, Vinyl Blue, Black
	M	48 cm	35 cm	
	L	53 cm	40 cm	
50 cm	S	44 cm	23 cm	
	M	49 cm	28 cm	
	L	54 cm	33 cm	
	XL	59 cm	38 cm	
60 cm	S	45 cm	22 cm	
	M	50 cm	27 cm	
	L	55 cm	32 cm	
	XL	60 cm	35 cm	

**3-part knee braces:**

**MECRON Knee Clinical**

Length	Angle	Colour
50 cm	0° / 20°	Schwarz-Gelb
60 cm	0° / 20°	

**MECRON Knee Pediatric**

Length	Colours
22 cm	Blau-Gelb,
32 cm	Blau-Orange,
42 cm	Blau-Hellblau

**MECRON Knee 3-part**

Length	Angle	Colours
40 cm	0° / 20°	Velours Blau,
50 cm	0° / 20°	Schwarz
60 cm	0° / 20°	

**MECRON Knee 3-part Plus**

Length	Angle	Colour
40 cm	0° / 20°	Schwarz
50 cm	0° / 20°	
60 cm	0° / 20°	

**Material:**

Outer material: vinyl, denim or velour  
Inner material: PU soft foam  
Textile covering: synthetic yarns  
Edging strips: 100% cotton  
Velcro-fasteners: polyamide  
Sewing threads: polyester  
Anti-slip nubs: silicone  
Carrying bag: PET (polyethylene terephthalate)

**Care:** The product is maintenance-free.

## Caution

Always follow these instructions for use. If you notice any side effects (e.g. swollen skin, pressure points, disrupted blood circulation, etc.) please consult your doctor. If used improperly, the product's function and optimum protection can no longer be guaranteed. This appliance is intended to assist you in your recovery and should only be used as prescribed by (or in consultation with) your doctor or medical specialist. DARCO (Europe) GmbH does not assume any responsibility.

Avoid using the brace in water and do not expose it to acids and solvents. It is advisable to wear dressings. We recommend that you refrain from driving while you are wearing this appliance. This aid is intended for single-patient use only.

## Cleaning instructions

All MECRON knee braces are washable at 30°. Make sure not to spin the braces. Remove the aluminium rods and close the velcro-fasteners prior to washing. Wash braces made of denim separately as the dye in this material may run when it is washed. After washing, pull the braces into shape, re-insert the rods and place horizontally to dry.

## Instructions for use for the following products:

### 1-part knee braces:

MECRON Knee Flexion | 20° bent knee brace according to Dr. Hildebrand

MECRON Knee Straight | 0° straight knee brace

### 3-part knee braces:

MECRON Knee 3-part | 0° or 20° 3-part universal knee brace

MECRON Knee 3-part Plus | 0° or 20° 3-part universal knee brace

MECRON Knee Clinical | 0° or 20° hospital brace

MECRON Knee Paediatric | 0° universal knee brace for children

The MECRON knee braces are used to immobilise your knee in the flexion position prescribed by your doctor. The immobilising knee braces feature medial and lateral aluminium rods to assist in lateral stabilization. The posterior aluminium rods are accommodated in bags with velcro fasteners. The rods are removable and can be re-bent as required. The braces feature velcro fasteners and thanks to their additional integrated foam pads provide a non-slip fit which is comfortable for patients. The materials used are easy on the skin and breathable.

## How to use 1-part knee braces:

1. Place the patient's leg into the immobilising knee brace. Position the patella recess to the left and the right alongside the kneecap.
2. Align the brace in such a way that the lateral rods lie along the knee's medial and lateral ligaments. Slightly bending the aluminium rods in the brace enables you to adjust them to the circumstances of the leg.
3. Next close the upper velcro strip first. Then, and only then, the other strips. Ensure to close the strips securely, but do not exert pressure. The brace must not be allowed to slide.

## How to use 3-part knee braces:

1. Separate the shell into 3 parts.
2. Place the patient's leg into the posterior part of the immobilising knee brace. Position the patella recess to the left and the right alongside the kneecap.
3. Afterwards take the two lateral parts and attach them to the posterior part of the immobilising knee brace in such a way that the lateral rods lie along the knee's medial and lateral ligaments. The markings enable the brace to be adjusted individually to the circumference of the patient's upper and lower and leg for a perfect fit (positioning aids supplied with the MECRON Knee 3-part Plus only). The patella recess of the three parts should be vertically aligned. Slightly bending the aluminium rods in the brace enables you to adjust them to the circumstances of the leg.
4. Next close the upper velcro strip first. Then, and only then, the other strips. Ensure to close the strips securely, but do not exert pressure. The brace must not be allowed to slide. The 3 individual parts enable you to achieve a secure fit.

You can stick the attached pad in the hollow of the knee area in the rail, if required (is only supplied with MECRON Knee 3-part). In certain indications the compression strip is intended for patella stabilisation (is only supplied with MECRON Knee 3-part / Plus) .

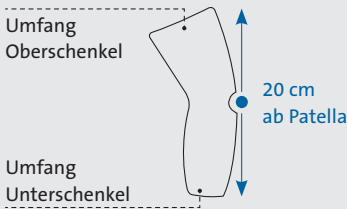
Attach the brace in such a way that it fits snugly on your body. Ensure not to trap your skin around the velcro fastener. To prevent constriction, do not over-tighten.

## Indications

- > Conservative therapy in case of collateral ligament injuries
- > Cruciate ligament instabilities
- > Meniscus injuries
- > Traumatic or degenerative changes to knee joints
- > Immobilisation after patella dislocation
- > Posttraumatic, postoperative immobilisation
- > Immobilisation after ligament injuries
- > Conditions after meniscus suture
- > Preoperative pain
- > Postoperative use for cartilage transplantation
- > Corrective osteotomy to straighten the mechanical axis of the leg

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