MECRON Shoulder Classic

MECRON Shoulder Universal

MECRON Shoulder Neopren

USA / GB Instructions for use

MECRON Shoulder Classic | shoulder bandage
MECRON Shoulder Universal | shoulder bandage
MECRON Shoulder Neopren | Schulterbandage

The MECRON shoulder bandages are for fixing the shoulder and immobilising the arm. The bandages consist of individual parts, the thorax strap, a cuff for the forearm and one for the wrist. This ensures secure shoulder and arm positioning, as prescribed by the doctor. The cut enables the bandages to be optimally adapted to the patient's anatomy. The bandage is suitable for long-term use.

Recommended use:

MECRON Shoulder Classic:

- Open the velcro strips of the shoulder bandage.
- Place the patient's forearm (i.e. wrist up to and including the elbow) at a 90-degree angle into the bandage.
- Close the two narrow straps at the back of the patient's neck.
- You can now fix the arm with the chest strap to prevent any rotary movement of the shoulder.

MECRON Shoulder Universal & MECRON Shoulder Neoprene:

- Take the removable cuffs off the shoulder bandage.
- Open the strap, place it around the patient's abdomen and close it.
- Now attach the loose cuffs to the abdominal strap.
- Open the velcro fasteners of the upper arm and wrist strap.
- Put the arm into the straps provided.
- Close the straps' velcro fasteners.
- Firmly attach the optional shoulder strap on the unaffected side to the abdominal strap both at the front and back to prevent the abdominal strap from slipping. (Only included with the MECRON Shoulder Universal)





Attach the bandages in such a way to ensure a snug fit. Ensure not to trap any skin around the velcro fasteners. To prevent constriction, do not over-tighten.

Caution

Always follow these instructions for use. If you notice any side effects (e.g. swollen skin, pressure points, disrupted blood circulation, etc.) please consult your doctor. If used improperly, the product's function and optimum protection can no longer be guaranteed.

This appliance is intended to assist you in your recovery and should only be used as prescribed by (or in consultation with) your doctor or medical specialist. DARCO (Europe) GmbH does not assume any responsibility.

Avoid using the bandage in water and exposure to acids and solvents. We recommend that you refrain from driving while you are wearing this appliance. This appliance is a single-use product.

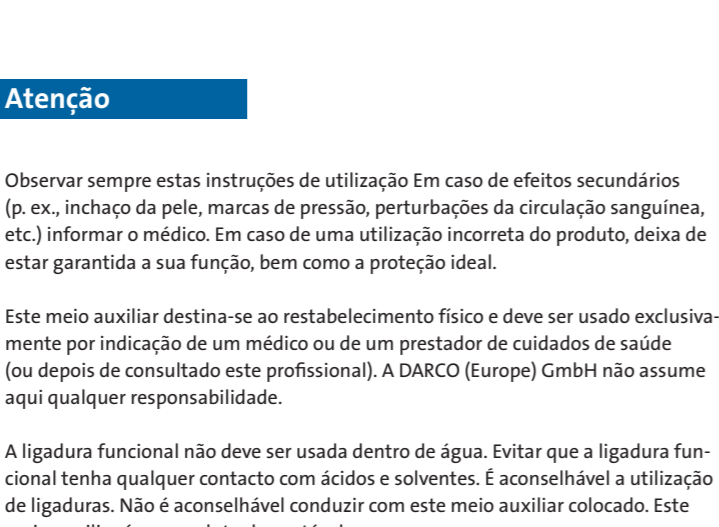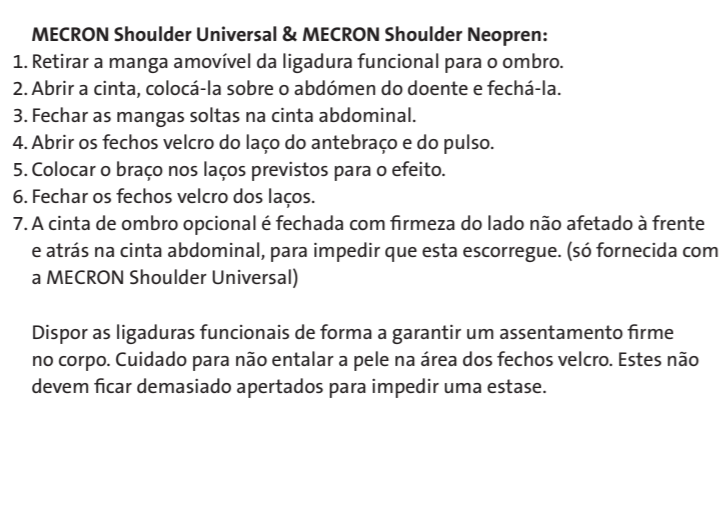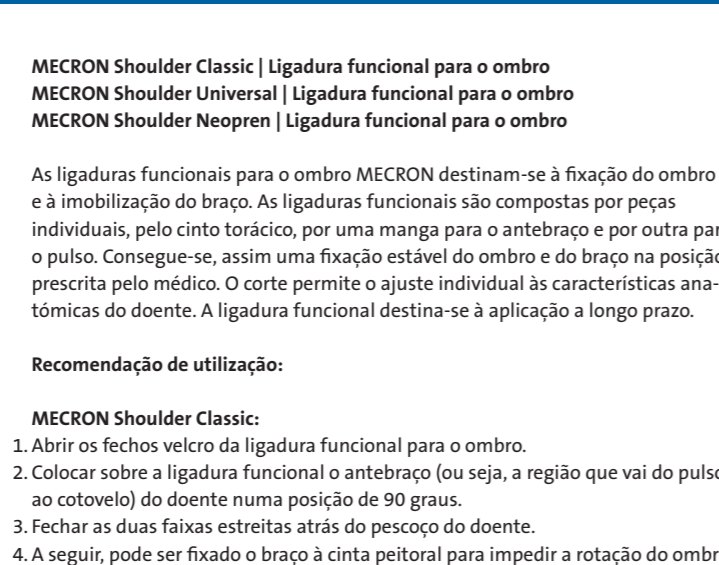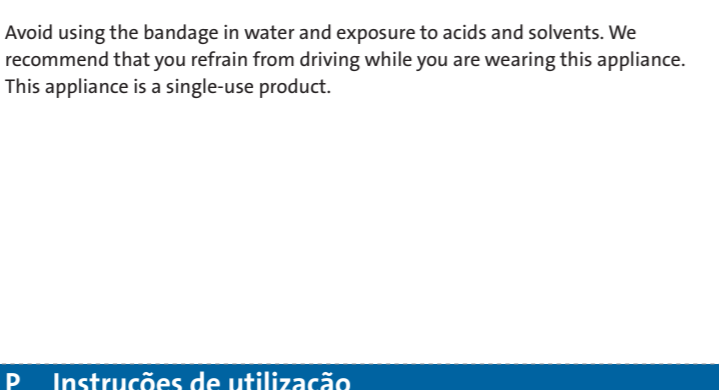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MECRON Shoulder Classic | Schulterbandage
MECRON Shoulder Universal | Schulterbandage
MECRON Shoulder Neopren | Schulterbandage

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