MECRON Wrist Guard

USA/GB Technical data

Sizes:
The wrist-forearm splint comes in 4 sizes depending upon the wrist's circumference. Measure around the wrist in a circle. The splint comes in versions for the right and left hand.

Size	Circumference
S	15 – 17 cm
M	17 – 19 cm
L	19 – 22 cm
XL	above 22 cm

Colour: blue vinyl, black

Material:
Outer side of the splint: PVC compound
Inner side of the splint: polyamide
Edging strip: 100% cotton
Velcro fasteners: polyamide

Care: The product is maintenance-free.

Caution

Always follow these instructions for use. If you notice any side effects (e.g. swollen skin, pressure points, disrupted blood circulation, etc.) please consult your doctor. If used improperly, the product's function and optimum protection can no longer be guaranteed.

This appliance is intended to assist you in your recovery and should only be used as prescribed by (or in consultation with) your doctor or medical specialist. DARCO (Europe) GmbH does not assume any responsibility.

Avoid using the splint in water and do not expose it to acids and solvents. It is advisable to wear dressings. We recommend that you refrain from driving while you are wearing this appliance. This appliance is a single-use product.

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MECRON
by **DARCO**

USA / GB Instructions for use

MECRON Wrist Guard
Wrist-forearm splint

The MECRON wrist-forearm splint is used to immobilize the wrist without restricting finger mobility. The wrist-forearm splint is stabilized by volar and dorsal aluminium splints. The splint is fixed with velcro fasteners and provides a non-slip fit which is comfortable for patients. The materials used are easy on the skin and breathable.

Recommended use:

1. Open the velcro strips of the wrist-forearm splint.
2. Place the patient's hand into the splint so that the thumb rests comfortably in the pre-defined recess.
3. First close the two velcro fasteners which are near to the wrist.
4. Now you can close the other velcro strips.

Attach the splint in such a way to ensure a snug fit. Ensure not to trap any skin around the velcro fasteners. To prevent constriction, do not over-tighten.

Indications

- > Acute / chronic wrist irritations
- > Arthralgias (degenerative, rheumatic)
- > Posttraumatic, postoperative immobilisation
- > Carpal tunnel syndrome
- > Partial paralyses

Contraindications:

- > No contraindications are known.

Cleaning instructions

The MECRON wrist-forearm splint is hand-washable. After washing, pull the splints into shape and place horizontally to dry.

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