Medburg Pro®

**USA/GB Technical data**

**Sizes:**  
3 Women sizes:  
WS (34 – 36,5), WM (37 – 38,5), WL (39 – 41)

4 Men sizes:  
MS (39 – 41), MM (41,5 – 43), ML (43,5 – 45), MXL (45 – 48)

**Colour:** Black  
Low and high version (extending the ankle)

**Material:**  
Outsole: TPR  
Insole: EVA  
Upper material: Foam Polyester Mesh  
Lining: Brushed Polyester  
Hook and loop fastener: Nylon

**Maintenance:**  
The product does not require maintenance.

**Construction features:**  
The upper part is made of a breathable nylon fabric which may be stretched lengthwise and crosswise, the rubber sole is made of TPR with shock absorbing effect. The sole profile enables an easier rolling-off movement.

**Cleaning instructions**

**Hand washing:**  
> When the shoe is slightly dirty, clean it with a damp cloth or a soft brush.  
> The shoe may not tumbled dry or dried on the heater. Allow the shoe to air dry at moderate temperatures.

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**MedSurg Pro®**

Post-op shoe

The MedSurg Pro® healing shoe provides stabilisation, shock absorbtion and offloading pressure after foot surgery. The shoe fits right and left equally.

1. Open the shoe by means of the Hook and loop fastener (A, B).
2. Please ensure that the foot is securely positioned in the shoe.
3. The shoe can be customized to the right or left foot by individually adjusting the Hook and loop fastener (A, B).
4. Pull the upper Hook and loop fastener A to the left or right side of the shoe so that the hook strap is located at the outside of the shoe on both sides. Place it over the flap and fasten accordingly. If necessary, please shorten the upper Hook and loop fastener individually.
5. The inner shoe flaps may overlap each other arbitrarily (C).
6. Close the shoe with the lower Hook and loop fastener strap B so that the DARCO Logo is located at the outside of the shoe.

**Indications**

> Postoperative for stabilisation and offloading, e. g. foot osteotomies

**Contraindication:**

> Open fractures

**Caution**

While wearing this shoe do not attempt to drive. Use extreme caution when climbing stairs (e. g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usually. Recommendation to wear bandages or socks in the shoe. This aid is intended for single-patient use only.

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