Nightsplint®

DARCO International, Inc.
810 Memorial Blvd.
Huntington, WV 25701, USA
Toll-free 800.999.8866
Phone 304.522.4883
Fax 304.522.0037
www.darcointernational.com

DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Telefon +49 (0) 88 07. 92 28-0 | Fax -22
info@darco.de, www.darco.de

Made in China

USA/GB Indication

- > Shortening of the plantar fascia
- > Achilles tendonitis
- > Prophylaxis of talipes equinus
- > Postoperative immobilization

Contraindication

- > Plantar flexion contracture exceeds 10°
- > No control with severe plantar flexion spasticity

Technical data

Sizes	Length	Height	Width
S 35 – 36	23.0 cm	33.0 cm	12.0 cm
M 37 – 39	23.5 cm	35.0 cm	13.0 cm
L 40 – 43	28.0 cm	37.0 cm	14.0 cm
XL 44 – 49	34.0 cm	38.0 cm	16.5 cm

Right and left fit.

Colour: Black

Material:
Backing splint: nylon
Liner: polyester foam
2 Heel wedges: EVA

Maintenance:
The product does not require maintenance. The DARCO NightSplint® splinting system is designed for the use of one person only.

Caution

This medical aid is designed to alleviate pain associated with tendon shortening and tendonitis in the heel area and is also used for treating Achilles tendonitis. This product is not intended for weight bearing for a longer period and is primarily designed for the use with bedbound patients. Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. If the product is handled incorrectly, its function and optimum protection can no longer be guaranteed. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example. This aid is intended for single-patient use only.

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Tamanhos	Comprimento	Altura	Largura
S 35 – 36	23,0 cm	33,0 cm	12,0 cm
M 37 – 39	23,5 cm	35,0 cm	13,0 cm
L 40 – 43	28,0 cm	37,0 cm	14,0 cm
XL 44 – 49	34,0 cm	38,0 cm	16,5 cm

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USA / GB Instructions for use

NightSplint® Splinting System

The NightSplint® splinting system is designed to stabilize dorsal extension which consequently results in a slight stretching of the Achilles tendon, the calf muscles as well as the plantar fascia. This promotes offloading the muscles and ligaments and encourages the process of healing.

- > two heel wedges for adjusting dorsal flexion
- > three hook and loop fastener straps for selective customization
- > smooth upper material – high comfort
- > non-slip sole

Recommendations for use:

Positioning the foot into the backing splint:

1. Open the hook and loop fastener straps at the front in the toe area underneath the splint. Remove the two heel wedges from the wedge pocket.
2. Enter the foot into the backing splint with the knee bended in order to realize the angle of maximum dorsiflexion.
3. Fix the belt across the instep with the hook and loop fastener strap in order to optimize the position of the heel.
4. Stretch the knee in order to check the fitting of the belt across the instep (If the heel is slipping inside the splint the belt is too loose, if the calf is squeezed in, the belt is too tight. Readjust and check again).
5. After checking the correct fit of the belt across the instep, the other two hook and loop fastener straps can be closed across ankle and shinbone in order to ensure a secure and comfortable fit.

Inserting the heel wedges into the backing splint:

1. Insert the heel wedges only under supervision of a physician or healthcare specialist.
2. With the foot in the splint open the strap located underneath the splint and initially insert one wedge thin end first.
3. Try and test this stabilizing orthosis for one hour. In case of skin irritation, changes in colour or increased pain symptoms, please contact your physician or healthcare specialist immediately.
4. If during the night no pain or discomfort occurs and your medical healthcare specialist agrees, the second wedge may be inserted the same way as the other.

Cleaning instructions

1. Remove the heel wedges.
2. Open the hook and loop fastener straps in the heel area of the product.
3. Take out the plastic body by bending together the two shell parts.
4. For cleaning use mild detergents only. The liner is only to be hand washed and air dried.

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