**USA/GB Indications**

- > Use after surgeries to correct hallux valgus, hammer toes, Tailor's bunions and other osteotomies
- > Treatment of diabetic ulcerations (forefoot)

**Contraindication:**  
> Open fractures

**Technical data**

**Sizes:**  
5 Adult sizes (OrthoWedge Light):  
XS (33,5 – 37), S (37,5 – 39,5), M (40 – 41,5), L (42,0 – 44,0), XL (44,5 – 47,0)

1 Children size (OrthoWedge Light Pediatric):  
Universal (up to size 31)

The shoe fits right and left equally.

**Material:**  
Outsole: EVA  
Insole: EVA  
Upper material: Foam Polyester (Mesh)  
Hook and loop fastener: Nylon

**Maintenance:**  
The product does not require maintenance.

**Construction features:**  
The inner sole can be removed and replaced by the PegAssist® Insole. This special shoe with a very light EVA sole is explicitly designed for usage in closed rooms.

**Recommended Add-ons:**  
**TwinShoe®**, Balance shoe  
**PegAssist® Insole**, Individually customizable insole for targeted off-loading

**Cleaning instructions**

**Hand washing:**  
> When the shoe is slightly dirty, clean it with a damp cloth or a soft brush.  
> The shoe may not tumbled dry or dried on the heater. Allow the shoe to air dry at moderate temperatures.

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#### USA/GB Instructions for use

**OrthoWedge Light / Pediatric**  
Shoe to reduce pressure on the forefoot

- Open the shoe (A).
- Insert the foot into the opened shoe.
- Make sure that the foot is positioned securely in the shoe.
- Close the shoe with the hook and loop fasteners (A). So that the fit is snug, but not too tight.
- Only roll as far as the inner edges (B) when walking. Do not roll-off as far as to the outer edges (C). Otherwise it may negatively influence the final outcome of surgery.
- When worn for longer periods, it is recommended that the height of the other foot is adapted to match (TwinShoe®). The shoe should not be worn for longer than 6 weeks.

#### Caution

#### USA/GB

Your doctor has prescribed a very high-quality shoe that has been developed and designed in such a way that less weight is applied to the part of the foot which has been operated on. This design makes it possible for ulcers to heal as rapidly as possible, after surgery, fractures, etc., whilst at the same time your mobility remains intact. Walking in this shoe is more difficult than in normal shoes; for this reason please read the following information:

You must not drive a vehicle while you are wearing this shoe. Be extremely careful when you climb stairs, cross a kerb and walk over uneven ground. Walk more slowly than usual. As you are not wearing a normal walking shoe, you need more ground clearance for the heel. Heavy weight (obese patients) can cause the functional shoe sole to break. If the shoe sole becomes worn, please consult your attending doctor or orthopaedic shoe technician. You are advised to wear surgical dressings or socks. If the product is not treated properly, its function and optimal protection are no longer guaranteed. This aid is intended for single-patient use only.

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