Puzzle Insole®
Modular Insole

The modular insole was developed for plantar pressure redistribution and is specifically designed for use after forefoot and midfoot surgery. By simply replacing the interchangeable modules, patients can achieve optimal pressure distribution depending on the indication. Additional pressure relief of up to 22% in the front area is achieved by the wedge-shaped sole that is significantly thinner in the heel area.

Recommendations for use:

- The black sole (fig. 1) serves as a carrier module. The modules of the blue sole (lower Shore hardness, fig. 2) are used for indication-related pressure distribution.
- Insert a blue module (narrow, wide and/or central) into the part of the black carrier sole where pressure needs to be relieved.

Possible combinations for the left foot:

- > little toe module (narrow), (fig. 3)
- > big toe module (wide), (fig. 4)
- > central module (fig. 5)

Possible combinations for the right foot:

- > little toe module (narrow), (fig. 6)
- > big toe module (big), (fig. 7)
- > central module (fig. 8)

- Remove the original sole from the shoe. (fig. 9)
- Place the adjusted Puzzle Insole® back into the shoe. (fig. 10) Ready to use.

Caution:

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. The function and optimal protection are no longer guaranteed if the product is incorrectly used.

> The modular insole can be worn on the right or left foot. Please make sure to use this product correctly!

The sole must always be complete (made up of all 4 parts) to ensure targeted pressure relief. Do not use the sole if a module is missing.

> It is recommended that dressings or socks are worn.
> The function and optimal protection are no longer guaranteed if the product is incorrectly used.

This aid, together with a special shoe, is intended to assist you in your recovery and should only be used as prescribed by (or in consultation with) your doctor or medical specialist. DARCO (Europe) GmbH does not assume any responsibility. It is recommended that dressings or socks are worn.

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Pressure due to indication

> simple exchanging of the insole modules

> various shoe-dualities in order to relieve pressure due to indication

> specifically designed for postoperative care after forefoot and metatarsal operations

> Lisfranc's joint line)

> Jones fracture (Puzzle module ends at the

> Tailor's bunion

> MTP-Module (small)

> Youngswick-Osteotomy (Puzzle module ends at Cuneiforme I)

> Lapidus-Arthrodese (TMT1-Arthrodese)

> Crescentic-Osteotomy

> Closing- and Opening-Wedge-Osteotomy

> Austin-, Scarf- and Akin-Osteotomy

> MTP-Arthrodese

> Large Toe Module (big)

> Metatarsalia II – IV)

(Puzzle module ends at the proximal end of

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