Puzzle Insole® | Modular Insole

Important notes: If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities. Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example. Do not drive a vehicle while you are wearing this device in a special shoe. Use extreme caution when walking with this device on uneven surfaces. Walk more slowly than usual. This device is intended for single-patient use only.

Purpose
The modular insole serves to redistribute plantar pressure and off-loading specific pressure zones. The sole is used especially after forefoot and midfoot surgery.

Indications / Contraindications

- Indications**
- Small Toe Module (narrow)
 - Tailor's bunion (digitus superductus)
 - Jones fracture (puzzle module extends to Lisfranc joint row)
- Large Toe Module (wide)**
- Metatarsophalangeal joint arthrodesis
 - Austin, Scarf and Akin osteotomy
 - Closing and opening wedge
 - Transverse and pendulum osteotomy
 - Lapidus arthrodesis (TMT1 arthrodesis)
 - Youngswick osteotomy (puzzle module extends to cuneiform I)
- Middle Module**
- Weil and / or Helal osteotomy I-IV
 - Morton's neuroma
 - Hammer toe correction
 - PIP / DIP arthrodesis
 - Metatarsal bone fractures II - IV
 - Shortening osteotomy of the metatarsal bones II - IV (puzzle module extends to proximal end of the metatarsal bones II - IV)
- Contraindications**
- Open fractures

Design features

The easy exchange of the modules allows an optimal, indication-dependent pressure distribution for the patient. Additional off-loading of 22% in the front area is achieved by the wedge-shaped sole, which becomes significantly thinner towards the heel area. The two EVA foam soles in black and blue differ only in the grade of Shore hardness.

Use

- The black sole (Fig. 1) serves as a support model. The blue sole modules (lower Shore hardness, Fig. 2) are used for indication-related pressure redistribution.
- Insert a blue module (narrow, wide and / or medium) into the black support sole where off-loading is to be provided. **Example combinations for the left foot:**
 - Small toe module (narrow, Fig. 3)
 - Big toe module (wide, Fig. 4)
 - Middle module (Fig. 5)**Example combinations for the right foot:**
 - Small toe module (narrow, Fig. 6)
 - Big toe module (wide, Fig. 7)
 - Middle module (Fig. 8)
- Remove the original sole from the shoe. (photo 9)
- Reinsert the adjusted Puzzle Insole® into the shoe. (Fig. 10) Finished.

Note: The sole must always be complete (made up of all 4 parts) to ensure targeted pressure relief. Do not use the sole if a module is missing.

Combination with other products

- The Puzzle Insole® product can be combined with the following DARCO products:
- MedSur®
 - MedSur® Pro®
 - Relief Dual®
 - Relief Dual® Plus
 - Relief Insert® Walker 2.0

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

Technical data

- Sizes**
- Three ladies sizes: WS (34 - 36.5), WM (37 - 38.5), WL (39 - 41)
 - Four mens sizes: MS (39 - 41), MM (41.5 - 43), ML (43.5 - 45), MXL (45.5 - 47)

The modular insole can be worn on the right or left foot. Please make sure to use this product correctly!

Colour Black-Blue
Material The two EVA foam soles in black and blue differ only in the Shore hardness. The blue sole has a lower Shore hardness (09) than the black sole (30).

Maintenance


The product does not require maintenance.

Cleaning instructions

The soles can be cleaned with lukewarm water and soap or with conventional disinfectants.

Disposal

Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

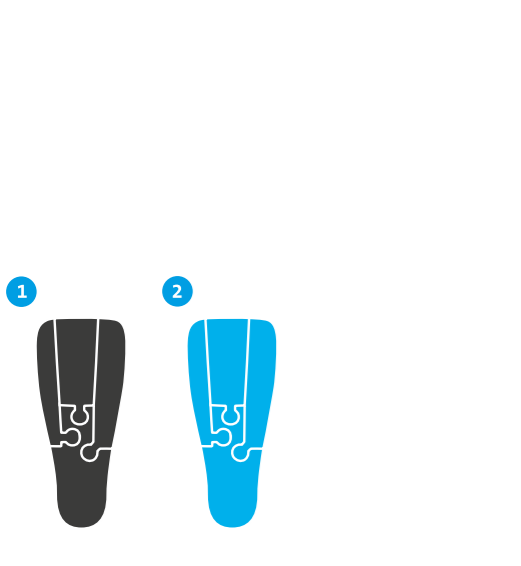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