Relief Contour Insole

Cushion insole

The Relief Contour Insole is a long-sole, cushion insole. Optimal pressure redistribution is achieved through the contours which are adapted to the anatomy of the foot. The special sole design leads to plantar weight-bearing of up to 90%. The self-supporting polyurethane material (PU) absorbs pressure peaks. The negative heel provides additional forefoot off-loading.

If needed, the PU material can be trimmed, which enables individual, patient-specific adjustment of the Relief Contour Insole.

Recommendations for use:

1. Remove the original sole from the shoe.
2. Insert the right or left Relief Contour Insole in the shoe in place of the original sole.
3. Note: This creates a right or left shoe from a shoe that can be worn on both sides.

Indication:

- > For use following hallux valgus and small toe surgeries (Austin, Akin, Scarf, Weil, Hohmann, etc.)
- > Talipes valgus, flat foot, splayfoot
- > Angioneuropathic changes of the foot in combination with mild foot deformities

Contraindications:

- > Open fractures
- > Not suitable for the treatment of the hind foot (negative heel)
- > Marked foot deformities of the diabetic foot

Instructions for cleaning:

The sole can be cleaned with lukewarm water or with conventional disinfectants. Do not dry the sole on a heating unit; instead allow it to dry at room temperature.

Technical data:

Sizes:

3 women's sizes:
WS (34 – 36.5), WM (37 – 38.5), WL (39 – 41)
4 men's sizes:
MS (39 – 41), MM (41.5 – 43), ML (43.5 – 45), MXL (45.5 – 47)

Right and left version available

Colour: Blue

Available in the corresponding sizes for the following aids:

- > Relief Dual® / Plus – Off-loading shoe
- > MedSurg – Healing shoe
- > MedSurg Pro® – Healing shoe (high shaft and low shaft)

Material:

The sole is made of polyurethane (PU).
The lining is made of 18% elastane and 82% polyamide (PA).

Maintenance: The product is maintenance-free.

Design features:

The Relief Contour Insole is constructed with a negative heel of 2.4 degrees in dorsal extension. This results in mild hind foot pressure and simultaneous forefoot off-loading. The transverse arch as well as medial and lateral longitudinal arch offer perfect pressure redistribution. Pressure peaks are reduced by the long-sole, continuous soft foot bed.

Caution:

Always follow these instructions for use. In case of side effects (e.g. skin swelling, pressure points, blood circulation disorders, etc.), please tell your doctor. If used improperly, the product's function and optimum protection can no longer be guaranteed. This appliance, together with a special shoe, is intended to assist you in your recovery and should only be used as prescribed by (or in consultation with) your doctor or medical specialist. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example. We recommend that you do not drive a vehicle while you are wearing this aid in a special shoe. Take extreme caution and go more slowly than usual. This aid is intended for single-patient use only.

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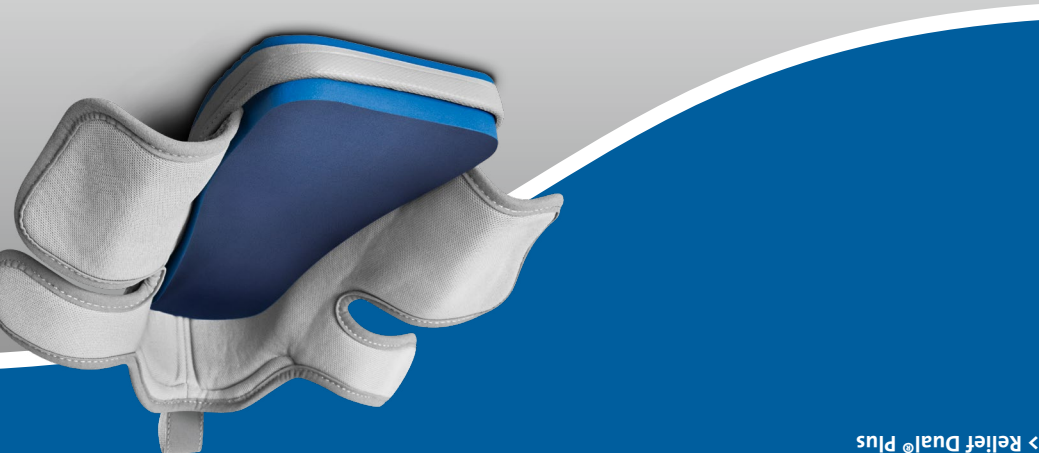
Made in EU

www.darco.de | www.darco-europe.com

REF: RC-M1-R	REF: RC-M2-R	REF: RC-M3-R	REF: RC-M4-R
left / links: <input type="checkbox"/> MS	left / links: <input type="checkbox"/> MM	left / links: <input type="checkbox"/> ML	left / links: <input type="checkbox"/> MXL
right / rechts: <input type="checkbox"/> MS	right / rechts: <input type="checkbox"/> MM	right / rechts: <input type="checkbox"/> ML	right / rechts: <input type="checkbox"/> MXL
REF: RC-W1-L	REF: RC-W2-L	REF: RC-W3-L	REF: RC-W4-L
left / links: <input type="checkbox"/> WS	left / links: <input type="checkbox"/> WM	left / links: <input type="checkbox"/> WL	left / links: <input type="checkbox"/> WXL
right / rechts: <input type="checkbox"/> WS	right / rechts: <input type="checkbox"/> WM	right / rechts: <input type="checkbox"/> WL	right / rechts: <input type="checkbox"/> WXL

Indication
> After hallux valgus and small toe surgeries (Austin, Akin, Scarf, Weil, Hohmann, etc.)
> Talipes valgus, flat foot, splayfoot
> Angioneuropathic changes of the foot in combination with mild foot deformities
> Optimal pressure redistribution is achieved through the contours which are adapted to the anatomy of the foot.
> The special sole design increases the plantar weight-bearing up to 90%.

Product features
> Cushion insole (long-sole)
> PU material can be trimmed
> Right and left versions available

Available for the following products in the according size:
> MedSurg
> MedSurg Pro®
> Relief Dual®
> Relief Dual® Plus

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