## USA / GB Technical data

### Sizes:

3 Women sizes:  
WS (34 – 36,5), WM (37 – 38,5), WL (39 – 41)

### 4 Men sizes:

MS (39 – 41), MM (41,5 – 43), ML (43,5 – 45), MXL (45,5 – 47)

**Colour:** Grey, Black

### Material:

Outsole: Rubber, EVA  
Insole: EVA  
Upper material: Polyester foam, Microfibre  
Inner liner: Polyester  
Edging strip: Cotton  
Hook and loop fastener: Nylon  
Adhesive: Polyurethane glue

**Maintenance:** The product does not require maintenance.

### Construction features:

The upper part is made of a breathable nylon fabric which may be stretched lengthwise and crosswise. The sole is made from EVA rubber with shock absorbing effect. The sole profile enables an easier rolling-off movement.

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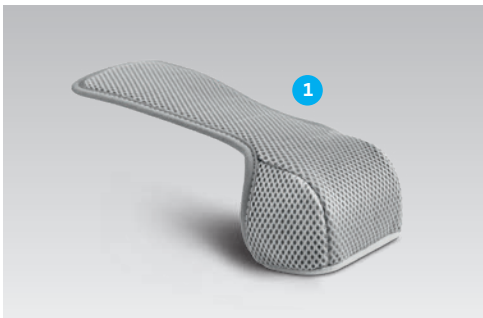
## USA/ GB Instructions for use

### Relief Dual® Off-loading Shoe

The Relief Dual® off-loading shoe provides stability and protects the foot against bumps and reduces pressure on it following foot surgery. The shoe fits right and left equally.

1. Open the shoe by means of the hook and loop straps (A, B).
2. Please ensure that the foot is securely positioned in the shoe.
3. The shoe can be customized to the right or left foot by individually adjusting the hook and loop straps (A, B).
4. Guide the upper hook and loop strap A through both straps on the shoe.
5. Fold the strap on the outer side (left or right) around the tab and secure it firmly. Close the shoe with the strap on the other side. If needed, the length of the upper hook and loop strap can be customized to fit.
6. The inner shoe tabs can optionally be placed over each other (C).
7. Close the shoe with the lower hook and loop fastener B such that the DARCO logo is on the outside of the shoe.

#### **Toe Cap (optional available):**

The DARCO Toe Cap (Fig. 1) allows the DARCO shoe to be worn in closed form. The cap protects the foot from cold and dirt and improves comfortable fitting.

## Indications

- > Postoperative for stabilisation and offloading, e. g. foot osteotomies

#### **Contraindication:**

- > Open fractures

## Cleaning instructions

#### **Hand washing:**

- > When the shoe is slightly dirty, clean it with a damp cloth or a soft brush.
- > The shoe may not tumbled dry or dried on the heater. Allow the shoe to air dry at moderate temperatures.

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