Walker

Relief Insert®

USA / GB Indication

> arthrodese of the first TMT joint (Lapudis arthrodeseis)
> In seguito a frattura di metatarso
> after hallux valgus surgery
> after hallux rigidus surgery
> after hammer and claw toe surgery
> fractures of the phalanges

> after surgery on the rearfoot
> for ligament rupture of the ankle joint

Contraindicazioni:
> upper ankle joint arthrodeseis
> acute Weber fractures (B, C)

Technical data

The lower leg / foot orthosis is available in seven sizes for the left and right foot.

Sizes:
3 women's sizes:
WS (34–36,5), WM (37–38,5), WL (39–41)

4 men's sizes:
MS (39–41), MM (41,5–43), ML (43,5–45), MXL (45,5–47)

Colour: Grau-Blue

Maintenance:
Das Produkt does not require maintenance.

Construction features

The Relief Insert® Walker consists of the following components:

Shoe:

The upper is made of breathable nylon mesh that can be stretched and adjusted lengthways and sideways. The outer sole is made of polyurethane with a shock absorbing effect. The sole profile facilitates the rolling off movement. The base of the shoe is reinforced throughout. Other features include the wide opening, individually adjustable fasteners, a removable sole as well as weatherproof durable material on the shank and sole.

Backing Splint:
The Relief Insert® consists of a hard shell portion (orthosis). The backing splint is characterised by a rigid footplate for pes equinus prophylaxis and guide rails for the medial and lateral stabilisation of the foot and lower leg region. The upper part of the hard shell part is attached with three

Cleaning instructions

- Hand wash shoe, backing splint, inlay and inner liner
- The insole can be cleaned with lukewarm water or with conventional disinfectants.

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USA / GB Instructions for use

Relief Insert® Walker Lower leg /foot orthosis

The Relief Insert® Walker is a lower leg /foot orthosis for immobilisation with simultaneous mobilisation...

Use as a backing splint Relief Insert®:

- 1. Open all four external hook and loop fasteners on the orthosis. To prevent the hook and loop fasteners from getting in your way, simply fasten them behind, back onto the respective fleece side.

Use as a lower leg /foot orthosis Relief Insert® Walker:

- 12. With the orthosis ready, let the patient step into the open Relief Dual® off-loading shoe from the front. Make sure that the insole is positioned securely at the front in the shoe.

Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. The function and optimal protection are no longer guaranteed if the product is incorrectly used.

or socks with the shoe /walker. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces.

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