**DARCO**  
**ROGG-Brace®**  
+ DARCO ROGG-Brace® Short

www.darco.de

DARCO (Europe) GmbH  
Gewerbegebiet 18  
82399 Raisting, Germany

Manufactured for:  
DARCO International, Inc.  
810 Memorial Blvd.  
Huntington, WV 25701, USA

Manufactured by:  
DARCO Int. Trading (Shanghai) LTD  
No. 128 Lane 3458 Hu Min Road  
Shanghai 201108, China

USA / GB Indication
> For optimal stabilization of the athlete's wrist
Contraindication
> Open fractures
Technical data
<p><b>Sizes:</b>  <b>DARCO ROGG-Brace®:</b> M (length 20.6 cm), L (length 22.1 cm)  <b>DARCO ROGG-Brace® Short:</b> Universal (length 12.5 cm)</p> <p>The splint is available for the right and left hand individually.</p> <p><b>Colour:</b> Black</p> <p><b>Material:</b>            Outer material: PVC            Lining: Polyester fabric            Elastic rubber bands: Isoprene            Multi-layer cotton jersey: Cotton elastic fabric</p> <p><b>Maintenance:</b>            The product does not require maintenance. The DARCO ROGG-Brace® is designed for the use of one person only.</p>

Cleaning instructions
The wrist splint is to be washed in the enclosed laundry bag. It is recommended to remove the inner palmar stay from the splint first and to close the velcro® strap. Please do not tumble dry, but air dry the splint.

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USA / GB Indication
> For optimal stabilization of the athlete's wrist
Contraindication
> Open fractures
Technical data
<p><b>Sizes:</b>  <b>DARCO ROGG-Brace®:</b> M (length 20.6 cm), L (length 22.1 cm)  <b>DARCO ROGG-Brace® Short:</b> Universal (length 12.5 cm)</p> <p>The splint is available for the right and left hand individually.</p> <p><b>Colour:</b> Black</p> <p><b>Material:</b>            Outer material: PVC            Lining: Polyester fabric            Elastic rubber bands: Isoprene            Multi-layer cotton jersey: Cotton elastic fabric</p> <p><b>Maintenance:</b>            The product does not require maintenance. The DARCO ROGG-Brace® is designed for the use of one person only.</p>
Cleaning instructions
The wrist splint is to be washed in the enclosed laundry bag. It is recommended to remove the inner palmar stay from the splint first and to close the velcro® strap. Please do not tumble dry, but air dry the splint.

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DARCO ROGG-Brace® / -Short  
Wrist Splint

The DARCO ROGG-Brace® wrist splint signifies optimal stabilization and protection of the wrist and has been developed in close cooperation with professional handball players. It is the ideal splint for field players and pitchers (with thumb loop). The bungee closure system provides a perfect fit together with a high compression. The DARCO ROGG-Brace® wrist splint is available in two versions: with or without thumb loop. The thumb loop and the palm area are especially designed to ensure optimum freedom for moving the fingers. The palmar stay made of plastic is included into the splint and additionally protects and stabilizes the wrist. The padded inner lining of the DARCO ROGG-Brace® wrist splint provides a perfectly snug fit.

#### Recommendations for use:

The bungee closure system enables a quick and easy application of this wrist splint by oneself. For maximum stabilization and a comfortable fit we recommend putting on the splint as follows:

1. Loosen the velcro® strap and widen the splint.
2. **DARCO ROGG-Brace® (with thumb loop), (figures A):** Apply the splint by leading the thumb completely through the thumb loop. **DARCO ROGG-Brace® Short (without thumb loop), (figures B):** Lead the hand completely through the splint until the lower part is finishing with the wrist.
3. Pull at the elastic bands by means of the velcro® strap and wrap them around the splint.
4. Test for a secure and comfortable fit of the splint and subsequently close the velcro® strap.
5. If required tighten the elastic rubber bands once more.

#### Caution

This product is intended for protecting and stabilizing the wrist during sports activities. Please consult your coach or physiotherapist for sensible use of the DARCO ROGG-Brace®. If you experience any pain, swelling, sensation changes, or any unusual reaction while using this product, discontinue use and contact your physician immediately.

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