# SlimLine®

# SlimLine® Pediatric

EN Cast Boot  
DE Gips- und Verbandschuh  
ES Zapato para escayola y postquirúrgico  
FR Chaussure pour plâtre et pansement  
IT Calzatura ortopedica e coprigesso  
NL Gips- en verbandschoen  
DA Gips- og bandagesko  
SV Gips- och förbandssko  
PL But na gips i opatrunek  
SL Mavčni in terapevtski čevlji  
HR Gips cipela i ortopedska cipela  
PT Sapato pós-operatório para gesso

# DARCO

## SlimLine®

## SlimLine® Pediatric

## EN Instructions for use

SlimLine® | Cast Boot

SlimLine® Pediatric | Pediatric Cast Boot

### Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This device assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a health-care professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example.

Do not drive a vehicle while you are wearing these shoes. Use extreme caution when walking with this device on uneven surfaces. Walk more slowly than usual.

This device is intended for single-patient use only.

## Purpose

The SlimLine® is a flat, versatile cast boot for the protection of all types of plaster casts and heavy compression dressings.

## Indications / Contraindications

### Indications

- > For plaster or cast treatments
- > For fractures
- > For postoperative care
- > Bulky dressings

### Contraindications

- > Open fractures

## Design features

### Adult and child model

The upper is made of a breathable EVA / polyester material. The shock-absorbing sole is made of TPR rubber. The sole profile facilitates the rolling off movement.

## Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

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DARCO International, Inc.
ALBO-Healthcare GmbH
V-M Orthotics Ltd.