(Toe Alignment Splint)
TAS®

DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Telefon +49 (0) 88 07.92 28-0 | Fax -22
info@darco.de | www.darco.de

Made in Germany

139807

USA/GB Technical data

Size: One size fits all, right and left fit
Colour: White

Material:

Elastic strap: Polyester, polyamide, viscose, elastane
Hook strap: Polyamide
Fleece strap: Polyamide
Velour: Polyamide
Label: Polyester satin
Label: Polyester satin
Elastic stabilisation strap: Polyester
Elastic heel strap: Polyester
Spring band steel: Spring band steel 14310
Sewing thread: Polyester

Maintenance: The product does not require maintenance.

Construction features:

The TAS® Toe Alignment Splint is a postoperative bandage for alignment that is used instead of taping. It enables positioning and alignment of toes in the correct position after surgery / correction, which has a decisive impact on the surgical result. A heel strap at the back of the foot reduces slippage of the midfoot bandage. A pleasant wearing comfort is ensured by the soft material and avoiding use of rigid plastic elements.

Instructions for use

TAS® (Toe Alignment Splint)

Toe Alignment Splint

The bandage is used for post-operative use to maintain the surgical result.

Hammer toe (fig. 1 – 3):

The elastic metatarsal band is designed so that it can be adjusted to the foot individually. Fasten the metatarsal band around the foot as seen in fig. 1 – 2.

The heel band prevents the bandage to change position and is applied as shown in fig. 3.

Tailor's bunion (fig. 4 – 5):

If necessary, the small toe loops might be fixed with a hook and loop strap to the metatarsal band (fig. 4 – 5).

Hallux valgus (fig. 6 – 9):

The big toe loop is secured as shown in fig. 6 – 7.

The stabilizing element is attached sideways to the big toe loop by means of a hook and loop strap (fig. 8 – 9).

Indications

Postoperatively after:

- > Hallux valgus surgery
- > Tailor's bunion surgery
- > Hammer toe surgery

Contraindication:

- > No contraindications are known.

Cleaning instructions

- > washable at 30° C

Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. This aid is intended for single-patient use only.

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