Function Comfort
TASO®

DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Telefon +49 (0) 88 07.92 28-0 | Fax -22
info@darco.de | www.darco.de

Made in Germany

USA/GB Technical data
6 Sizes: 35 – 36, 37 – 38, 39 – 40, 41 – 42, 43 – 44, 45 – 46
Packaging unit: 1 pair with tension band
Colour: Black
Material: 90% cotton, 8% polyamide, 2% polyurethane, Latex free
Maintenance: The product does not require maintenance.
Construction features: The hallux valgus corrective sock has separate boxes for the great toe and small toes. In addition, an elasticated strap is integrated on the outside of the sock for correction of the great toe. The elasticated band inside the sock or foot begins at the distal phalanx of the great toe and runs along the side of the inner ball towards the heel. An elasticated tension band runs along the outer side. It starts behind the outer ball of the fifth MTP and finishes in the heel area on the outer side below the lateral malleolus. The sock can be worn during the day in shoes. It is important that shoes have sufficient room in the toe area.

Cleaning instructions

- > Wash in the washing machine (at 60 degrees) – turn sock inside out before washing
- > Do not dry in dryer, do not bleach, do not iron, do not dry-clean

Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. The function and optimal protection are no longer guaranteed if the product is incorrectly used. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. This aid is intended for single-patient use only.

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TASO® Function Comfort
Hallux Valgus Corrective Sock

The sock can be used preventively in early-stage hallux valgus. The corrective sock also has a high therapeutic effect when worn post-operatively.

1. Gather together the sock from top to bottom using your fingers and slip your foot into the sock. Place your toes in the toe boxes. Please slip your great toe and small toes slowly into the toe separators.
2. Next, pull the sock backwards over the heel and then upwards over the inner and outer ankle joint until the sock sits on the foot without wrinkles.
3. Check that that the heel is seated correctly and then check the position of the tension band. Only in this way is satisfactory correction of the great toe ensured.

With this application, you correct the hallux valgus in motion and prevent it moving further outwards into the malposition due to the ground contact pressure. It has a high therapeutic effect when used while walking. Pay particular attention in the first few days of wearing the socks. Should worsening pain occur in the first metatarsophalangeal joint, discontinue wearing the socks immediately and gradually increase use over a period of 1 – 2 weeks. In this way, you mobilise the capsule of the first metatarsophalangeal joint slowly and the joint is not overstrained. Discuss the exact procedure with your doctor.

You can also wear the socks at night for passive correction, thus mobilising the first metatarsophalangeal joint in a gentle way.

Indications

- > **Preventively:** Prevent development or progression of the malposition
- > **Postoperatively:** After hallux valgus surgery to protect the surgical correction

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