Function Control
TASO®

USA / GB Technical data

6 Sizes:
35 – 36, 37 – 38, 39 – 40, 41 – 42, 43 – 44, 45 – 46

Packaging unit: 1 sock with strap, 1 sock without strap

Colour: Black

Material: 90% cotton, 8% polyamide, 2% polyurethane, Latex free

Maintenance: The product does not require maintenance.

Construction features:
The hallux valgus corrective sock with strap has a separate boxes for the great and small toes. In addition, a strap for correction of the great toe is attached to the medial side at the height of the first DIP joint of the great toe. This strap runs from the great toe towards the heel. The strap is fixed below the medial malleolus by a hook and loop fastener only when the desired correction of the great toe has been achieved. The strap is then guided under the foot and is fixed using the hook and loop fastener at approx. the height of the fifth metatarsal joint. The sock can be worn during the day in shoes. It is important that shoes have sufficient room in the toe area.

Cleaning instructions

- > Wash in the washing machine (at 60 degrees) – turn sock inside out before washing
- > Do not dry in dryer, do not bleach, do not iron, do not dry-clean

Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. The function and optimal protection are no longer guaranteed if the product is incorrectly used. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. This aid is intended for single-patient use only.

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USA/ GB Instructions for use

TASO® Function Control

Hallux Valgus Corrective Sock with Strap

The sock with strap is a postoperative aid after hallux valgus surgery. However, the sock can also be worn as a preventive measure.

- Gather together the sock from top to bottom using your fingers and slip your foot into the sock. Place your toes in the toe boxes. Please slip your great toe and small toes slowly into the toe separators.
- Next, pull the sock backwards over the heel and then upwards over the inner and outer ankle joint until the sock sits on the foot without wrinkles.
- Check that the heel is seated correctly.
- Correct the great toe individually using the loose strap and fasten the strap securely to the inner side of the sock.
- Now place the strap around the heel and faster the strap to the outer hook and loop fastener fabric.

With this application, you correct the hallux valgus in motion and prevent it moving further outwards into the malposition due to the ground contact pressure. It has a high therapeutic effect when used while walking. Pay particular attention in the first few days of wearing the socks. Should worsening pain occur in the first metatarsophalangeal joint, discontinue wearing the socks immediately and gradually increase use over a period of 1 – 2 weeks. In this way, you mobilise the capsule of the first metatarsophalangeal joint slowly and the joint is not overstrained. Discuss the exact procedure with your doctor.

You can also wear the socks at night for passive correction, thus mobilising the first metatarsophalangeal joint in a gentle way.

Indications

- > **Postoperatively:** After hallux valgus surgery to protect the surgical correction
- > **Preventively:** Prevent development or progression of the malposition

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