USA / GB Technical data

5 Sizes:
XS (33,5 – 37), S (37,5 – 39,5), M (40 – 41,5), L (42 – 44), XL (44,5 – 47)

The shoe fits right and left equally.

Material:
Outsole: TPR
Insole: EVA
Upper material: Polyester

Maintenance:
The product does not require maintenance.

Construction features:
The upper material consists of breathable polyester material. The rubber sole is made of TRP with shock absorbing effect. The sole profile enables an easier rolling-off movement.

Can be used with the following DARCO products:
OrthoWedge /-Light
HeelWedge®
Body Armor® Walker II (in the case of club foot position of 120 degrees)
WCS™

Cleaning instructions

Hand washing:

- > When the shoe is slightly dirty, clean it with a damp cloth or a soft brush.
- > Hand washing: The TwinShoe may be hand washed in cold water with mild soap and may be air dried afterwards. Please do not use strong solvents for cleaning (alcohol etc.).
- > The shoe may not tumbled dry or dried on the heater. Allow the shoe to air dry at moderate temperatures.

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5 Größen:
XS (33,5 – 37), S (37,5 – 39,5), M (40 – 41,5), L (42 – 44), XL (44,5 – 47)

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Made in China:

DARCO Int. Trading (Shanghai) LTD
No. 128 Lane 3458 Hu Min Road
Shanghai 201108, China

USA / GB Instructions for use

TwinShoe
Balance shoe

- Open the boot (A).
- Enter into the opened boot.
- Please ensure that the foot is securely positioned in the boot and that the heel area of the foot is as far back as possible in the boot.
- Close shoe with the velcro® straps (A). The foot should be placed comfortably in the boot, not too loose and not too tight.

Indications

- > Balance shoe for postoperative care following forefoot and midfoot surgery.
- > When prescribing aids which cause a potential difference in leg lengths (such as a forefoot off-loading shoe).
- > The TwinShoe helps avoid pelvic tilt and can also be worn outside of enclosed spaces. The length of time this aid is worn depends on the indication.

Contraindication:

- > Open fractures

Caution

Your physician has prescribed you a very unusual looking shoe. Walking with this boot is more difficult than walking with conventional footwear; please read the following precautions.

Do not attempt to drive while wearing this boot. Use extreme caution when climbing stairs, stepping up onto curbs and walking on uneven surfaces. Walk more slowly than usual. You are not wearing a normal street shoe and more clearance is required for the wedge sole. Excessive weight bearing (edematous patients) may cause a breakdown of the shoe's functional sole. In case the sole must be fixed due to extensive wearing, please contact your physician or orthopedic technician. Recommendation to wear with bandages or socks.

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