DARCO International, Inc.  
810 Memorial Blvd.  
Huntington, WV 25701, USA  
Toll-free 800.999.8866  
Phone 304.522.4883  
Fax 304.522.0037  
www.darcointernational.com

DARCO (Europe) GmbH  
Gewerbegebiet 18  
82399 Raisting, Germany  
Telefon +49 (0) 88 07 92 28-0 | Fax -22  
info@darco.de | www.darco.de

Made in China

## USA/GB Indication

- > For treatment of diabetic foot syndrome
- > For healing of open wounds and pressure ulcers and ulcerations
- > For pressure offloading
- > Postoperative after toe and forefoot surgery

### Contraindication

- > Open fractures

### Technical data

#### Size:

The shoe is available in seven sizes for the right and left foot.

#### Material:

Outsole: PU  
Upper material: Natural leather

#### Maintenance:

The product does not require maintenance. The WCS® is designed for the use of one person /patient only.

#### Plantar layer system:

Material	Density	Thickness	Durometer*
EVA	soft	6,3 mm	26
EVA	medium	6,3 mm	42
EVA	firm	6,3 mm	50
Insole foam	high	3,1 mm	29

\*The insole durometers may vary within 2° to 3° of those listed.

### Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. It is advisable to wear dressings or socks with the shoe. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This aid is intended for single-patient use only.

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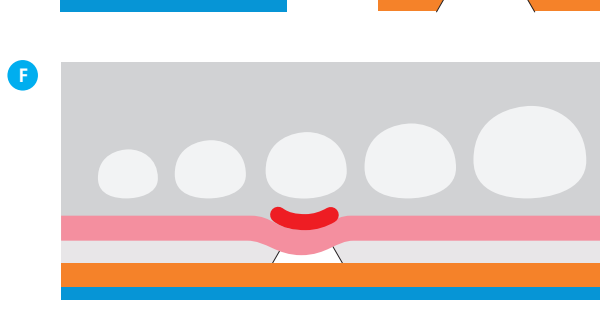
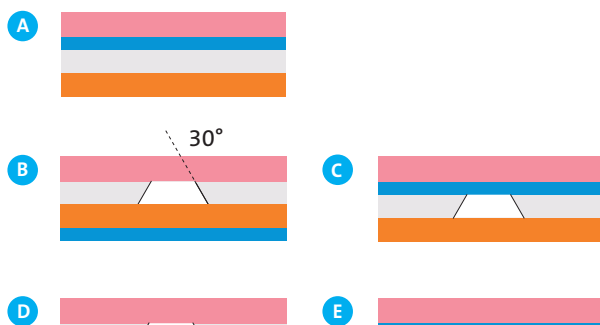
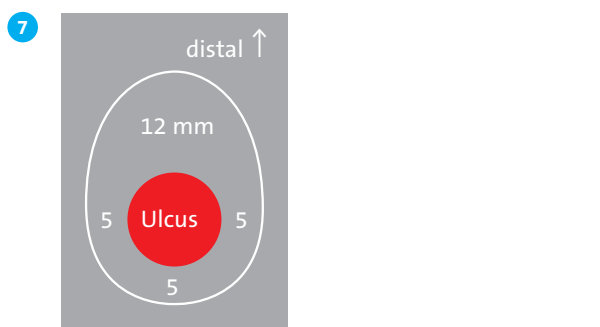
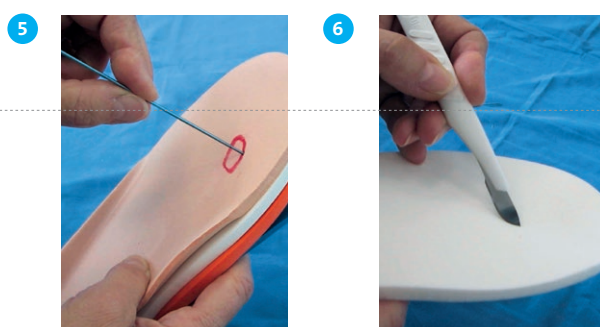
### WCS®

Wound care shoe

The WCS® Wound Care Shoe is available individually for the right and left foot. Determining the correct shoe size is important for the perfect functionality of the shoe. On the inside of the sole you will find a »D« which should be placed distally just in front of the first metatarsal head. This will ensure that the pressure point of the rocker sole will be located distally to the metatarsal head. The hook and loop closure system enables to easily and comfortably open and close the wound care shoes. The WCS® system in particular supports patients limited in their mobility. For a proper stabilization of the foot the hook and loop straps should be closed accordingly. The proven removable insole system with its 4 layers (fig. 2) is already included in the shoe (fig. 1) or enclosed with delivery and can be individually adjusted (fig. 6).

#### Marking the ulceration and customizing the insoles:

- Place a sterile film over the wound or pressure ulcer (fig. 3) and draw a circle on the film around the affected area with the marker pen supplied (fig. 4).
- The patient should now stand up in the shoe/s onto the insoles which then pictures the ulcer onto the upper layer of the insole.
- Puncture the two layers following the upper layer by penetrating this specific layer with a sharp tool. Then mark the area to be cut out (fig. 5).
- Draw the areas to be removed onto the lower side of the layers as described in fig. 7.

#### Recommendations for use:

- The Wound Care Shoe is mainly used for the treatment of open and closed ulcerations and ulcers (Malum perforans). The period of application depends on the grade of ulceration.
- Daily wear with indoor and outdoor use.

#### Insole cutting:

- No plantar open ulcers
- Level of ulceration 0 and 1, Wagner Scale, open wounds, bevel cut with approx. 30°
- After wound healing (of B)
- Level of ulceration 2 and up, Wagner Scale, open wound
- After wound healing (of D)
- Cross section depicting offloading lesion under third metatarsal head

## Cleaning instructions

When slightly dirty clean the WCS® shoes with a damp cloth or a soft brush. Please do not use strong solvents for cleaning (alcohol etc.). Do not tumble dry or on the stove. Allow to air dry. The enclosed insoles can be washed with warm water and soap or cleaned with regular disinfectants.

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