MECRON Elastic Tape/Strong

MECRON Elastic Tape

MECRON Elastic Tape Strong

MECRON

USA / GB Instructions for use

MECRON Elastic Tape / Strong

Elastic Tape for Kinesiology

- > A clear overview of indication and disposition is required.
- > Find out whether the patient shows allergic reaction to taping material.
- > Measure the required tape length.
- > Round off the corners of the piece of tape.
- > Remove body hair around application area (optional). Please use a long-hair trimmer (no wet shaving!).
- > Apply the tape on clean (with warm water and soap) and dry skin.
- > When applying a base, attach the base with no tension, if possible.
- > Ensure that ends are laid down with no tension.
- > Ensure a correct body posture of the patient when applying the tape.
- > Attach the tape with the outside of the thumb, making sure that the tape does not arch and that the tape is not creased.
- > Avoid touching the glue layer during use to prevent removal of the adhesive.
- > After attachment of the tape rub it on, to activate the glue and prolong the adhesion. Take care that you are rubbing the tape along its length as indicated by the lymphatic and muscle techniques.
- > When removing the tape, fix the surrounding tissue with your hand. This reduces your patient's pain and protects the skin from excessive irritation.
- > The MECRON Elastic Tape / Strong is a single-use product.

Indications

- > for treatment of muscular pain
- > to improve performance by means of continuous stimulation
- > stimulates the blood circulation

Contraindications:

- > large and serious skin injuries
- > open wounds with risk of infection
- > nondistinctive diagnoses
- > allergic reactions to tape material
- > pulmonary veno-occlusive disease
- > thrombovascular disease
- > in the event of cancer or tumors

Technical data

Size: 5 cm × 5 m

Colours: MECRON Elastic Tape: blue, red, yellow, black, beige / Strong: pink, light blue

Material: Cotton and acrylic adhesive, Latex free

Maintenance: The product does not require maintenance.

Caution

Always follow these instructions for use. Contact your doctor if you sense any unusual reaction (such as skin swelling, skin redness etc.). In case of improper handling function and maximum safety of the product can no longer be guaranteed. The clinical efficacy of kinesiology tapes is professionally controversial and not scientifically proven by studies.

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DARCO (Europe) GmbH