# MECRON Elastic Tape/Strong

MECRON Elastic Tape

MECRON Elastic Tape Strong

---

# MECRON

---

## USA / GB Instructions for use

### MECRON Elastic Tape / Strong

Elastic Tape for Kinesiology

- > A clear overview of indication and disposition is required.
- > Find out whether the patient shows allergic reaction to taping material.
- > Measure the required tape length.
- > Round off the corners of the piece of tape.
- > Remove body hair around application area (optional). Please use a long-hair trimmer (no wet shaving!).
- > Apply the tape on clean (with warm water and soap) and dry skin.
- > When applying a base, attach the base with no tension, if possible.
- > Ensure that ends are laid down with no tension.
- > Ensure a correct body posture of the patient when applying the tape.
- > Attach the tape with the outside of the thumb, making sure that the tape does not arch and that the tape is not creased.
- > Avoid touching the glue layer during use to prevent removal of the adhesive.
- > After attachment of the tape rub it on, to activate the glue and prolong the adhesion. Take care that you are rubbing the tape along its length as indicated by the lymphatic and muscle techniques.
- > When removing the tape, fix the surrounding tissue with your hand. This reduces your patient's pain and protects the skin from excessive irritation.
- > The MECRON Elastic Tape / Strong is a single-use product.

## Indications

- > for treatment of muscular pain
- > to improve performance by means of continuous stimulation
- > stimulates the blood circulation

### Contraindications:

- > large and serious skin injuries
- > open wounds with risk of infection
- > nondistinctive diagnoses
- > allergic reactions to tape material
- > pulmonary veno-occlusive disease
- > thrombovascular disease
- > in the event of cancer or tumors

## Technical data

**Size:** 5 cm × 5 m

**Colours:** MECRON Elastic Tape: blue, red, yellow, black, beige / Strong: pink, light blue

**Material:** Cotton and acrylic adhesive, Latex free

**Maintenance:** The product does not require maintenance.

## Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. If the product is handled incorrectly, its function and optimum protection can no longer be guaranteed. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. This device is a single-use product. The clinical efficacy of kinesiology tapes is professionally controversial and not scientifically proven by studies.

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TS Co., Ltd.
Emergo Europe B.V.