# MECRON Pre-Tape

**MECRON**

## USA / GB Instructions for use

### **MECRON Pre-Tape**

#### Foam Underwrap

- > To begin with, obtain a clear picture of the condition of the patient being treated.
- > Find out whether the patient shows allergic reaction to taping material.
- > Apply the underwrap to clean and dry skin.
- > Ensure a correct body posture of the patient when applying the tape.
- > Make sure that the underwrap does not arch and that the tape is not creased.
- > Many tapes are stretched during application but ensure that measured force is used.  
The tissue (circulation) must not be constricted.
- > Keep moulding the tape so that it fits correctly.
- > The MECRON Pre-Tape is a single-use product.

## Indications

- > as cushioning to protect the skin in taping
- > to position padding under a cast

#### **Contraindications:**

- > large and serious skin injuries
- > open wounds with risk of infection
- > nondistinctive diagnoses
- > allergic reactions to tape material

## Technical data

**Size:** 7 cm × 27 m

**Colour:** Blue

**Material:** Polyurethane (PU foam), Latex free

**Maintenance:** The product does not require maintenance.

## Caution

Always follow these instructions for use. Contact your doctor if you sense any unusual reaction (such as skin swelling, skin redness etc.). In case of improper handling function and maximum safety of the product can no longer be guaranteed.

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**Manufactured by:**

DARCO Int. Trading (Shanghai) LTD  
No. 128 Lane 3458 Hu Min Road  
Shanghai 200020, China

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DARCO (Europe) GmbH