# MECRON Pre-Tape

**MECRON**

## USA / GB Instructions for use

### MECRON Pre-Tape

#### Foam Underwrap

- > To begin with, obtain a clear picture of the condition of the patient being treated.
- > Find out whether the patient shows allergic reaction to taping material.
- > Apply the underwrap to clean and dry skin.
- > Ensure a correct body posture of the patient when applying the tape.
- > Make sure that the underwrap does not arch and that the tape is not creased.
- > Many tapes are stretched during application but ensure that measured force is used. The tissue (circulation) must not be constricted.
- > Keep moulding the tape so that it fits correctly.
- > The MECRON Pre-Tape is a single-use product.

## Indications

- > as cushioning to protect the skin in taping
- > to position padding under a cast

### Contraindications:

- > large and serious skin injuries
- > open wounds with risk of infection
- > nondistinctive diagnoses
- > allergic reactions to tape material

## Technical data

**Size:** 7 cm × 27 m

**Colour:** Blue

**Material:** Polyurethane (PU foam), Latex free

**Maintenance:** The product does not require maintenance.

## Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. If the product is handled incorrectly, its function and optimum protection can no longer be guaranteed. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. This aid is a single-use product.

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