
# MECRON Standard Tape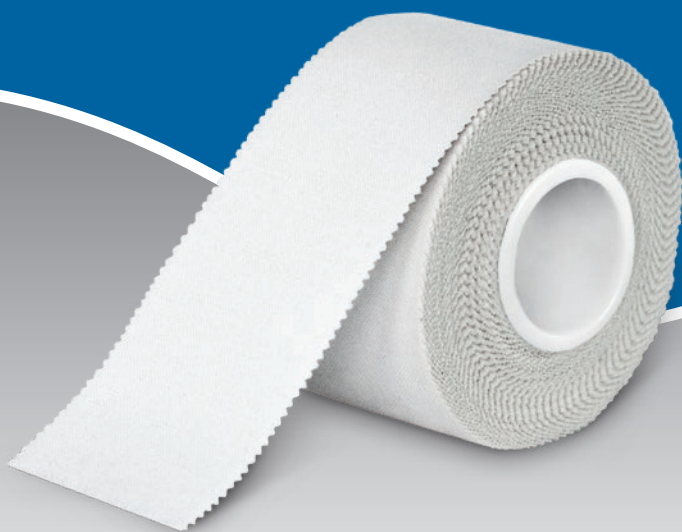

**MECRON**

## USA / GB Instructions for use

### **MECRON Standard Tape**

Non-elastic Athletic Tape

- > A clear overview of indication and disposition is required.
- > Find out whether the patient shows allergic reaction to taping material.
- > Remove body hair around application area (optional). Please use a long-hair trimmer (no wet shaving!).
- > Apply the tape on clean and dry skin.
- > Ensure a correct body posture of the patient when applying the tape.
- > Make sure that the tape does not arch and that the tape is not creased.
- > Many tapes are stretched during application but ensure that measured force is used. The tissue (circulation) must not be constricted.
- > Keep moulding the tape so that it fits correctly.
- > When removing the tape, fix the surrounding tissue with your hand. This reduces your patient's pain and protects the skin from excessive irritation.
- > The MECRON Standard Tape is a single-use product.

## Indications

- > for treatment of muscular pain
- > for stabilization of joints and muscles

### **Contraindications:**

- > large and serious skin injuries
- > open wounds with risk of infection
- > nondistinctive diagnoses
- > allergic reactions to tape material

## Technical data

**Size:** 3.8 cm × 10 m

**Colour:** White

**Material:** Cotton, Zinc oxide adhesive and Latex

**Maintenance:** The product does not require maintenance.

## Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. If the product is handled incorrectly, its function and optimum protection can no longer be guaranteed. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. This aid is intended for single-patient use only.

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miro Verbandstoffe GmbH