
MECRON Standard Tape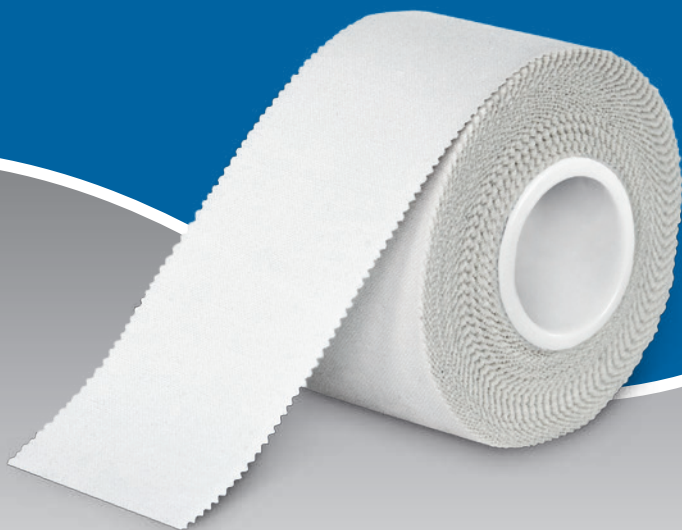

MECRON

USA / GB Instructions for use

MECRON Standard Tape

Non-elastic Athletic Tape

- > A clear overview of indication and disposition is required.
- > Find out whether the patient shows allergic reaction to taping material.
- > Remove body hair around application area (optional). Please use a long-hair trimmer (no wet shaving!).
- > Apply the tape on clean and dry skin.
- > Ensure a correct body posture of the patient when applying the tape.
- > Make sure that the tape does not arch and that the tape is not creased.
- > Many tapes are stretched during application but ensure that measured force is used. The tissue (circulation) must not be constricted.
- > Keep moulding the tape so that it fits correctly.
- > When removing the tape, fix the surrounding tissue with your hand. This reduces your patient's pain and protects the skin from excessive irritation.
- > The MECRON Standard Tape is a single-use product.

Indications

- > for treatment of muscular pain
- > for stabilization of joints and muscles

Contraindications:

- > large and serious skin injuries
- > open wounds with risk of infection
- > nondistinctive diagnoses
- > allergic reactions to tape material

Technical data

Size: 3.8 cm × 10 m

Colour: White

Material: Cotton, Zinc oxide adhesive and Latex

Maintenance: The product does not require maintenance.

Caution

Always follow these instructions for use. Contact your doctor if you sense any unusual reaction (such as skin swelling, skin redness etc.). In case of improper handling function and maximum safety of the product can no longer be guaranteed.

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DARCO (Europe) GmbH